

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN 1100 AGENCY GOODYEAR POLICE DEPT
DATE 8-8-14 TIME 1244
INTOXILYZER SERIAL # 80-001858 LOCATION 1111 S LITCHFIELD RD GOODYEAR AZ

- (K) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 097 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan 1100

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

08/08/2014

12:44:06

Standard Lot#: 32012100A1

Last Changed By: W. WILL #974

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	12:44:36
0.100 Cal Check	0.097	12:44:57
Air Blank	0.000	12:45:27

Preventative Maintenance
Performed -- Timer Reset

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

08/08/2014

12:38:43

Standard Lot#: 32012100A1

Last Changed By: W. WILL #974

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	12:39:44
0.100 Cal Check	0.097	12:40:05
Air Blank	0.000	12:40:35
Air Blank	0.000	12:43:03

*Input Timed Out

DID NOT
COMPLETE TEST
IN TIME
WILL/100