

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST W.T. RYAN #1100 AGENCY GOODYEAR P.D.

DATE 5-1-14 TIME 1604

INTOXILYZER SERIAL # 80-000312 LOCATION 1111 S LITCHFIELD RD GOODYEAR AZ

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.

2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.

2. Error recognition logic system functioning.

Not a Successfully Completed Test Sequence printed

3. Proper sample recognition system.

Not a Successfully Completed Test Sequence printed

Deficient sample printed.

() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

REMOVED FROM DSP NO PROBLEMS

SIGNATURE W.T. Ryan #1100

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

05/01/2014

16:04:32

Standard Lot#: 32012100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 05/01/1959

Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	16:06:50
Diagnostic Test	Pass	16:07:25
Air Blank	0.000	16:07:52
0.100 Cal Check	0.099	16:08:13
Air Blank	0.000	16:08:43
Subject Test	0.000	16:09:05
Air Blank	0.000	16:09:32
Wait	ABT*	
Air Blank	0.000	16:10:08

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 1111 S LITCHFIELD RD
Serial Number: 80-000312
Core Version: 8105.48
05/01/2014 16:10:39

Standard Lot#: 32012100A1
Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST
DOB: 05/01/1959
Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	16:11:30
Air Blank	0.000	16:11:59

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

05/01/2014

16:12:23

Standard Lot#: 32012100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 05/01/1959

Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	16:13:08
Air Blank	RFI*	16:13:17

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 1111 S LITCHFIELD RD
Serial Number: 80-000312
Core Version: 8105.48
05/01/2014 16:14:17

Standard Lot#: 32012100A1
Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST
DOB: 05/01/1959
Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	16:15:20
Diagnostic Test	Pass	16:15:55
Air Blank	0.000	16:16:22
0.100 Cal Check	0.099	16:16:43
Air Blank	0.000	16:17:13
Subject Test	0.035*	16:20:20
Air Blank	0.000	16:20:53
Air Blank	0.000	16:21:20
Subject Test	RGE**	16:21:36
Air Blank	0.000	16:22:17

*Deficient Sample
**Range Exceeded

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

05/01/2014

16:23:05

Standard Lot#: 32012100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 05/01/1959

Sex: M

Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	16:24:06
Diagnostic Test	Pass	16:24:41
Air Blank	0.000	16:25:08
0.100 Cal Check	0.099	16:25:29
Air Blank	0.000	16:25:59
Subject Test	0.022*	16:29:07
Air Blank	0.000	16:29:38
Air Blank	0.000	16:30:06
Subject Test	XXX**	16:30:20
Air Blank	0.000	16:30:54

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

05/01/2014

16:33:05

Standard Lot#: 32012100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test g/210L Time

Air Blank 0.000 16:33:36
0.100 Cal Check 0.098 16:33:56
Air Blank 0.000 16:34:27

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN 1100 AGENCY GOODYEAR P.D.
DATE 5-1-14 TIME 1602
INTOXILYZER SERIAL # 80-000312 LOCATION 1111 S LITCHFIELD RD GOODYEAR AZ

- (S) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (S) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (S) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (S) 4. Air blank completed.
- (S) 5. Calibration check completed. Test results 0.099 AC.
- (S) 6. Air blank completed.
- (S) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan 1100

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

05/01/2014

16:02:17

Standard Lot#: 32012100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test g/210L Time

Air Blank 0.000 16:03:00
0.100 Cal Check 0.099 16:03:21
Air Blank 0.000 16:03:51

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W. T. Payer 1100 AGENCY GOODYEAR P.D.
DATE 5-23-14 TIME 1111
INTOXILYZER SERIAL # 80-000312 LOCATION 1111 S LITCHFIELD RD GOODYEAR AZ

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 099 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W. T. Payer 1100

INTOXILYZER 8000
Location: 1111 S LITCHFIELD RD
Serial Number: 80-000712
Core Version: 8105.48
05/23/2014 11:11:03

Standard Lot#: 32012100A1
Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	11:11:37
0.100 Cal Check	0.099	11:11:58
Air Blank	0.000	11:12:28

Preventative Maintenance
Performed -- Timer Reset