

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Wiu AGENCY Goodyear PD  
DATE 9/6/2008 TIME 1533  
INTOXILYZER SERIAL # 80-000312 LOCATION Goodyear PD

- (W) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution \_\_\_\_\_ AC.  
OR  
Pour a standard alcohol concentration solution .100 AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (W) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (W) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (W) 4. Air blank completed.
- (W) 5. Calibration check completed. Test results 0. 103 AC.
- (W) 6. Air blank completed.
- (W) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: GOODYEAR PD

Serial Number: 80-000312

09/06/2008

15:33:14


-----  
Standard Lot#: 804301I

Last Changed By: J. TRAINING #99999

-----  
QAS: W. WILL #974

GOODYEAR PD  
-----

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | 0.000  | 15:33:42 |
| Cal Check | 0.103  | 15:34:00 |
| Air Blank | 0.000  | 15:34:30 |

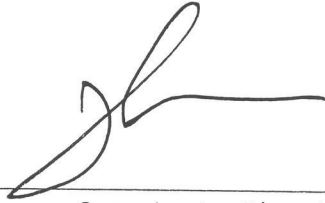

\_\_\_\_\_  
Operator's Signature