

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST W.T. RYAN 1100 AGENCY GOODYEAR P.D.

DATE 2-17-15 TIME 1419

INTOXILYZER SERIAL # 80-000312 LOCATION 1111 S LITCHFIELD RD GOODYEAR AZ

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC.
- 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

NO PROBLEMS

SIGNATURE W.T. Ryan 1100

INTOXILYZER 8000  
Location: 1111 S LITCHFIELD RD  
Serial Number: 80-000312  
Core Version: 8105.48  
02/17/2015 14:19:35

---

Standard Lot#: 25913100A2  
Last Changed By: W. RYAN #1100

---

QAS: W. RYAN #1100  
GOODYEAR PD

---

Operator: W. RYAN #1100  
GOODYEAR PD

---

Subject: TEST,TEST  
DOB: 02/17/1959  
Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	14:20:34
Diagnostic Test	Pass	14:21:09
Air Blank	0.000	14:21:37
0.100 Cal Check	0.100	14:21:58
Air Blank	0.000	14:22:28
Subject Test	0.000	14:23:02
Air Blank	ABT*	14:23:21
Air Blank	0.000	14:23:49

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: 1111 S LITCHEFIELD RD  
Serial Number: 80-000312  
Core Version: 8105.48  
02/17/2015 14:24:17

---

Standard Lot#: 25913100A2  
Last Changed By: W. RYAN #1100

---

QAS: W. RYAN #1100  
GOODYEAR PD

---

Operator: W. RYAN #1100  
GOODYEAR PD

---

Subject: TEST,TEST  
DOB: 02/17/1959  
Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	14:24:59
Air Blank	0.000	14:25:27

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: 1111 S LITCHFIELD RD  
Serial Number: 80-000312  
Core Version: 8105.48  
02/17/2015 14:25:48

---

Standard Lot#: 25913100A2  
Last Changed By: W. RYAN #1100

---

QAS: W. RYAN #1100  
GOODYEAR PD

---

Operator: W. RYAN #1100  
GOODYEAR PD

---

Subject: TEST,TEST  
DOB: 02/17/1959  
Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	14:26:37
Air Blank	RFI*	14:26:45

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: 1111 S LITCHFIELD RD  
Serial Number: 80-000312  
Core Version: 8105.48  
02/17/2015 14:28:32

---

Standard Lot#: 25913100A2  
Last Changed By: W. RYAN #1100

---

QAS: W. RYAN #1100  
GOODYEAR PD

---

Operator: W. RYAN #1100  
GOODYEAR PD

---

Subject: TEST,TEST  
DOB: 02/17/1959  
Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	14:29:42
Diagnostic Test	Pass	14:30:17
Air Blank	0.000	14:30:44
0.100 Cal Check	0.101	14:31:05
Air Blank	0.000	14:31:35
Subject Test	0.096*	14:34:43
Air Blank	0.000	14:35:15
Air Blank	0.000	14:35:42
Subject Test	RGE**	14:36:08
Air Blank	0.000	14:36:52

\*Deficient Sample

\*\*Range Exceeded

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: 1111 S LITCHFIELD RD  
Serial Number: 80-000312  
Core Version: 8105.48  
02/17/2015 14:38:42

---

Standard Lot#: 25913100A2  
Last Changed By: W. RYAN #1100

---

QAS: W. RYAN #1100  
GOODYEAR PD

---

Operator: W. RYAN #1100  
GOODYEAR PD

---

Subject: TEST,TEST  
DOB: 02/17/1959  
Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	14:39:38
Diagnostic Test	Pass	14:40:13
Air Blank	0.000	14:40:40
0.100 Cal Check	0.100	14:41:01
Air Blank	0.000	14:41:31
Subject Test	0.533*	14:44:39
Air Blank	0.000	14:45:12
Air Blank	0.000	14:45:39
Subject Test	RGE**	14:46:23
Air Blank	0.000	14:47:01

\*Deficient Sample

\*\*Range Exceeded

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: 1111 S LITCHFIELD RD  
Serial Number: 80-000312  
Core Version: 8105.48  
02/17/2015 14:50:05

---

Standard Lot#: 25913100A2  
Last Changed By: W. RYAN #1100

---

QAS: W. RYAN #1100  
GOODYEAR PD

---

Operator: W. RYAN #1100  
GOODYEAR PD

---

Subject: TEST,TEST  
DOB: 02/17/1959  
Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	14:51:08
Diagnostic Test	Pass	14:51:43
Air Blank	0.000	14:52:10
0.100 Cal Check	0.099	14:52:31
Air Blank	0.000	14:53:01
Subject Test	0.118*	14:56:09
Air Blank	0.000	14:56:41
Air Blank	0.000	14:57:08
Subject Test	XXX**	14:57:22
Air Blank	0.000	14:57:54

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

02/17/2015

14:58:57

-----  
Standard Lot#: 25913100A2

Last Changed By: W. RYAN #1100

-----  
QAS: W. RYAN #1100

GOODYEAR PD

-----  
Operator: W. RYAN #1100

GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	14:59:27
0.100 Cal Check	0.100	14:59:47
Air Blank	0.000	15:00:18

Preventative Maintenance  
Performed -- Timer Reset



EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN 1100 AGENCY GOODYEAR POLICE DEPT  
DATE 2-17-15 TIME 1416  
INTOXILYZER SERIAL # 80-000312 LOCATION 1115 LITCHFIELD RD GOODYEAR AZ

(✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(✓) 4. Air blank completed.  
(✓) 5. Calibration check completed. Test results 0.101 AC.  
(✓) 6. Air blank completed.  
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan 1100

INTOXILYZER 8000  
Location: 1111 S LITCHFIELD RD  
Serial Number: 80-000312  
Core Version: 8105.48  
02/17/2015 14:16:03

-----  
Standard Lot#: 25913100A2  
Last Changed By: W. RYAN #1100  
-----

QAS: W. RYAN #1100  
GOODYEAR PD  
-----

Operator: W. RYAN #1100  
GOODYEAR PD.  
-----

Test	g/210L	Time
Air Blank	0.000	14:16:44
0.100 Cal Check	0.101	14:17:05
Air Blank	0.000	14:17:35

Preventative Maintenance  
Performed -- Timer Reset