

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST W.T. RYAN #1100 AGENCY GOODYEAR P.D.

DATE 5-8-15 TIME 2129

INTOXILYZER SERIAL # 80-001858 LOCATION 1111 S LITCHFIELD RD GOODYEAR AZ

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES WM NO _____

COMMENTS

NO ISSUES

SIGNATURE W.T. Ryan #1100

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

05/08/2015

21:29:57

Standard Lot#: 25913100A2

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 05/08/1959

Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:31:22
Diagnostic Test	Pass	21:31:57
Air Blank	0.000	21:32:25
0.100 Cal Check	0.099	21:32:45
Air Blank	0.000	21:33:15
Subject Test	0.000	21:33:30
Air Blank	0.000	21:33:58
Wait	ABT*	
Air Blank	0.000	21:34:55

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 1111 S LITCHFIELD RD
Serial Number: 80-001858
Core Version: 8105.48
05/08/2015 21:35:50

Standard Lot#: 25913100A2
Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST
DOB: 05/08/1959
Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	21:36:29
Air Blank	0.000	21:36:57

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

05/08/2015

21:38:45

Standard Lot#: 25913100A2

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 05/08/1959

Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	21:39:25
Air Blank	0.000	21:39:53

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 1111 S LITCHFIELD RD
Serial Number: 80-001858
Core Version: 8105.48
05/08/2015 21:40:10

Standard Lot#: 25913100A2
Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST
DOB: 05/08/1959
Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:41:05
Diagnostic Test	Pass	21:41:40
Air Blank	0.000	21:42:07
0.100 Cal Check	0.099	21:42:28
Air Blank	0.000	21:42:57
Subject Test	0.075*	21:46:05
Air Blank	0.000	21:46:36
Air Blank	0.000	21:47:03
Subject Test	XXX**	21:48:39
Air Blank	0.000	21:49:12

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

05/08/2015

21:50:54

Standard Lot#: 25913100A2

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	21:51:25
0.100 Cal Check	0.099	21:51:45
Air Blank	0.000	21:52:15

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN #1100 AGENCY GOODYEAR P.D.

DATE 5-8-15 TIME 2126

INTOXILYZER SERIAL # 80-001858 LOCATION 1111 S LITCHFIELD RD GOODYEAR AZ

(✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

(✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

(✓) 4. Air blank completed.

(✓) 5. Calibration check completed. Test results 0. 098 AC.

(✓) 6. Air blank completed.

(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan 1100

DPS Form Exh G-2 (Rev 05-01)

NO ISSUES

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

05/08/2015

21:26:30

Standard Lot#: 25913100A2

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test g/210L Time

Air Blank 0.000 21:27:41
0.100 Cal Check 0.098 21:28:02
Air Blank 0.000 21:28:31

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST W.T. RYAN 1100 AGENCY GOODYEAR P.D.
DATE 5-29-15 TIME 1328
INTOXILYZER SERIAL # 80-001858 LOCATION 1111 S LITCHFIELD RD GOODYEAR AZ

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

NO PROBLEMS

SIGNATURE

W.T. Ryan 1100

INTOXILYZER 8000
Location: 1111 S LITCHFIELD RD
Serial Number: 80-001858
Core Version: 8105.48
05/29/2015 13:23:07

Standard Lot#: 25913100A2
Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST
DOB: 05/29/1959
Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:24:08
Diagnostic Test	Pass	13:24:43
Air Blank	0.000	13:25:11
0.100 Cal Check	0.099	13:25:31
Air Blank	0.000	13:26:01
Subject Test	0.000	13:26:15
Air Blank	0.000	13:26:43
Wait	ABT*	
Air Blank	0.000	13:27:19

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 1111 S LITCHFIELD RD
Serial Number: 80-001858
Core Version: 8105.48
05/29/2015 13:27:38

Standard Lot#: 25913100A2
Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST
DOB: 05/29/1959
Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	13:28:17
Air Blank	RFI**	13:28:35

*Improper Sample
**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 1111 S LITCHFIELD RD
Serial Number: 80-001858
Core Version: 8105.48
05/29/2015 13:29:06

Standard Lot#: 25913100A2
Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST
DOB: 05/29/1959
Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	13:29:43
Air Blank	0.000	13:30:12

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 1111 S LITCHFIELD RD
Serial Number: 80-001858
Core Version: 8105.48
05/29/2015 13:30:54

Standard Lot#: 25913100A2
Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST
DOB: 05/29/1959
Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:31:48
Diagnostic Test	Pass	13:32:23
Air Blank	0.000	13:32:51
0.100 Cal Check	0.099	13:33:11
Air Blank	0.000	13:33:41
Subject Test	0.068*	13:36:48
Air Blank	0.000	13:37:20
Air Blank	0.000	13:37:47
Subject Test	XXX**	13:38:26
Air Blank	0.000	13:39:00

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

05/29/2015

13:39:34

Standard Lot#: 25913100A2

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test g/210L Time

Air Blank 0.000 13:40:04
0.100 Cal Check 0.099 13:40:25
Air Blank 0.000 13:40:55

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN 1100 AGENCY GOODYEAR P.D.

DATE 5-29-15 TIME 1320

INTOXILYZER SERIAL # 80-001858 LOCATION 1111 S LITCHFIELD RD GOODYEAR AZ

(S) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (S) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (S) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (S) 4. Air blank completed.
- (S) 5. Calibration check completed. Test results 0. 100 AC.
- (S) 6. Air blank completed.
- (S) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan 1100

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

05/29/2015

13:20:41

Standard Lot#: 25913100A2

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	13:21:12
0.100 Cal Check	0.100	13:21:32
Air Blank	0.000	13:22:02

Preventative Maintenance
Performed -- Timer Reset