## EXHIBIT G-2 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

## ARIZONA DEPARTMENT OF PUBLIC SAFETY

## STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

## STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W. T. RYAN #1100 AGENCY GOODYGAR P.D.
DATE 6-27-15 TIME 1/25
INTOXILYZER SERIAL # 80 - 001 858 LOCATION [1] S LITCHFIED B GOOD
1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution OR
Pour a standard alcohol concentration solutionAC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER
4. Air blank completed. 5. Calibration check completed. Test results 0. 098 AC. 6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.
SIGNATURE W-T. Reger 1100
OPS Form Exh G-2 (Rev 05-01)

NO PRUBLEMS NOTED

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858 Core Version: 8105.48

06/27/2015

11:25:26

11:26:48

-----

Standard Lot#: 25913100A2

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

-----

Operator: W. RYAN #1100

GOODYEAR PD

Preventative Maintenance Performed -- Timer Reset

Air Blank 0.000