

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT

DATE 10-1-16 TIME 1530 HRS

INTOXILYZER SERIAL # 80-000312 LOCATION 1111 S LITCHFIELD RD GOODYEAR AZ

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.101 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan #1100

DPS Form Exh G-2 (Rev 05-01)

DID NOT PRINT RESULTS RECEIVED
PRINTED AT DPS CRIME LAB

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number : 80-000312

Core Version: 8105.48

10/01/2016

15:30:14

Standard Lot#: 26614100A6

Last Changed By: W. RYAN

1100

QAS: W. RYAN

1100

GOODYEAR PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 15:30:56 |
| 0.100 Cal Check | 0.101 | 15:31:17 |
| Air Blank | 0.000 | 15:31:47 |

W. T. Ryan #1100

Operator's Signature

Reprinted From Database

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT

DATE 10-1-16 TIME 1551 HRS

INTOXILYZER SERIAL # 80-000312 LOCATION 1111 S LITCHFIELD RD GOODYEAR AZ

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 101 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan #1100

DPS Form Exh G-2 (Rev 05-01)

DID NOT PRINT RESULTS RECORD.
PRINTED AT DPS CRIME LAB.

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number : 80-000312

Core Version: 8105.48

10/01/2016

15:51:35

Standard Lot#: 26614100A6

Last Changed By: W. RYAN # 1100

QAS: W. RYAN # 1100

GOODYEAR PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 15:52:06 |
| 0.100 Cal Check | 0.101 | 15:52:27 |
| Air Blank | 0.000 | 15:52:57 |

W. T. Ryan 1100

Operator's Signature

Reprinted From Database

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT

DATE 10-1-16 TIME 2011 HRS

INTOXILYZER SERIAL # 80-000312 LOCATION 1111 S LITCHFIELD RD GOODYEAR AZ

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

4. Air blank completed.

5. Calibration check completed. Test results 0. 101 AC.

6. Air blank completed.

7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan #1100

DPS Form Exh G-2 (Rev 05-01)

DID NOT PRINT RESULTS RECORDS
PRINTED AT DPS, ^{INTOX} TAKEN OUT OF
SERVICE

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number : 80-000312

Core Version: 8105.48

10/01/2016

20:11:11

Standard Lot#: 26614100A6

Last Changed By: W. RYAN

1100

QAS: W. RYAN

1100

GOODYEAR PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 20:11:41 |
| 0.100 Cal Check | 0.101 | 20:12:02 |
| Air Blank | 0.000 | 20:12:32 |

W. T. Ryan #1100

Operator's Signature



ARIZONA DEPARTMENT OF PUBLIC SAFETY

INTOXILYZER MAINTENANCE REPORT

Page 1 of 1

AGENCY Goodyear Police Department
Goodyear, AZ 85338
OFFICER RYAN, #1100
DATE October 12, 2016



INSTRUMENT

80000312

INSTRUMENT LOCATION

Goodyear Police Department

MAINTENANCE REQUESTED

Repair

MAINTENANCE PERFORMED

Parts Replaced:

Could not duplicate. Instrument functioning properly.

A handwritten signature in black ink, appearing to read "Joseph M Slowinski".

JOSEPH M SLOWINSKI, #6101, Electronics Technician
Central Regional Crime Laboratory
2102 W. Encanto Blvd., Phoenix, AZ 85009
602-223-2159

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN 1100 AGENCY GOODYEAR POLICE DEPT

DATE 10-26-16 TIME 0004

INTOXILYZER SERIAL # 80-006312 LOCATION 1111 S LITCHFIELD RD GOODYEAR AZ

- (M) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (V) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (M) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (W) 4. Air blank completed.
- (M) 5. Calibration check completed. Test results 0.101 AC.
- (M) 6. Air blank completed.
- (M) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan #1100

DPS Form Exh G-2 (Rev 05-01)

STANDARD CHANGE COMPLETED

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

10/26/2016

00:04:28

Standard Lot#: 14816100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 00:04:59 |
| 0.100 Cal Check | 0.101 | 00:05:20 |
| Air Blank | 0.000 | 00:05:50 |

Preventative Maintenance
Performed + Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST W. T. RYAN 1100 AGENCY GOODYEAR POLICE DEPT
DATE 10-26-16 TIME 0018 HRS
INTOXILYZER SERIAL # 80-000312 LOCATION 1111 SLITCHFIELD RD GOODYEAR AZ

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

BROUGHT BACK INTO SERVICE
WOULD NOT PRINT CHANGED PRINTERS

SIGNATURE W. T. Ryan 1100

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

10/26/2016

00:13:29

Standard Lot#: 14816100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 10/26/1959

Sex: M Weight: 180

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 00:14:30 |
| Diagnostic Test | Pass | 00:15:05 |
| Air Blank | 0.000 | 00:15:32 |
| 0.100 Cal Check | 0.099 | 00:15:53 |
| Air Blank | 0.000 | 00:16:23 |
| Subject Test | 0.000 | 00:16:39 |
| Air Blank | 0.000 | 00:17:06 |
| Wait | ABT* | |
| Air Blank | 0.000 | 00:17:45 |

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

10/26/2016

00:18:09

Standard Lot#: 14816100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 10/26/1959

Sex: M Weight: 180

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | IPS* | 00:18:50 |
| Air Blank | 0.000 | 00:19:19 |

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

10/26/2016

00:19:39

Standard Lot#: 14816100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 10/26/1959

Sex: M Weight: 180

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | RFI* | 00:20:31 |
| Air Blank | 0.000 | 00:20:59 |

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 1111 S LITCHFIELD RD
Serial Number: 80-000312
Core Version: 8105.48
10/26/2016 00:21:20

Standard Lot#: 14816100A1
Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST
DOB: 10/26/1959
Sex: M Weight: 180

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 00:22:19 |
| Diagnostic Test | Pass | 00:22:54 |
| Air Blank | 0.000 | 00:23:21 |
| 0.100 Cal Check | 0.099 | 00:23:42 |
| Air Blank | 0.000 | 00:24:12 |
| Subject Test | 0.052* | 00:27:19 |
| Air Blank | 0.000 | 00:27:51 |
| Air Blank | 0.000 | 00:28:18 |
| Subject Test | XXX** | 00:28:44 |
| Air Blank | 0.000 | 00:29:17 |

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

10/26/2016

00:29:50

Standard Lot#: 14816100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test g/210L Time

Air Blank 0.000 00:30:22
0.100 Cal Check 0.099 00:30:43
Air Blank 0.000 00:31:13

Preventative Maintenance
Performed -- Timer Reset