

INTOXILYZER 8000

Location: 11 S 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

02/08/2017

22:48:46

Standard Lot#: 14816100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test g/210L Time

Air Blank 0.000 22:49:17
0.100 Cal Check 0.100 22:49:38
Air Blank 0.000 22:50:08

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT
DATE 2-8-17 TIME 2248 HRS

INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145TH AVE GOODYEAR AZ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (PS) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (PS) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (PS) 4. Air blank completed.
- (PS) 5. Calibration check completed. Test results 0. 100 AC.
- (PS) 6. Air blank completed.
- (PS) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan #1100

DPS Form Exh G-2 (Rev 05-01)

NO PROBLEMS