

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT  
DATE 5-29-17 TIME 0138  
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145TH AVE GOODYEAR AZ

- (S) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (S) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (S) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (S) 4. Air blank completed.
- (S) 5. Calibration check completed. Test results 0. 098 AC.
- (S) 6. Air blank completed.
- (S) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan 1100

W. T. Ryan #1100  
0138

IN LATE 1940s  
0100

W. T. Ryan #1100  
2-27-17  
80-00018

OPS

W. T. Ryan  
1100

A

11/27/17

INTOXILYZER 8000

Location: 11 S 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

05/29/2017

01:38:43

-----  
Standard Lot#: 14816100A1

Last Changed By: W. RYAN #1100

-----  
QAS: W. RYAN #1100

GOODYEAR PD

-----  
Operator: W. RYAN #1100

GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	01:39:23
0.100 Cal Check	0.098	01:39:44
Air Blank	0.000	01:40:14

Preventative Maintenance  
Performed -- Timer Reset

*W. Ryan*