

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. Genthalt AGENCY GOODYEAR PD
DATE 6/2/17 TIME 1458
INTOXILYZER SERIAL # 80-001258 LOCATION GPD

- (R) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution, 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (R) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (R) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (R) 4. Air blank completed.
- (R) 5. Calibration check completed. Test results 0. 099 AC.
- (R) 6. Air blank completed.
- (R) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE VA 1144

2/24/17

INTOXILYZER 8000
Location: 11 S 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
06/02/2017

14:58:41

Handwritten mark

Standard Lot#: 28515100A1
Last Changed By: W. RYAN #1100

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	14:59:11
0.100 Cal Check	0.099	14:59:32
Air Blank	0.000	15:00:01

Preventative Maintenance
Performed -- Timer Reset

Handwritten mark

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE


QA SPECIALIST M. Gu, Ibaul AGENCY GOODYEAR PD
DATE 6/2/17 TIME 1501
INTOXILYZER SERIAL # 80-001858 LOCATION 6PD

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1144

mem 3

INTOXILYZER 8000
Location: 11 S 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
06/02/2017

15:01:41 

Standard Lot#: 23016100A1
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	15:02:11
0.100 Cal Check	0.100	15:02:32
Air Blank	0.000	15:03:01

Preventative Maintenance
Performed -- Timer Reset





ISO/IEC 17025:2005 Accredited Laboratory

Certificate of Analysis

Certificate ID: 9481
Part #: BAC105L100T
Cylinder Size: 105L
Lot Number: 23016100A1
Expiration: 9/5/2018

0.100 BAC (For the calibration of instruments used to determine breath alcohol concentration)

Contents: 105 Liters @ 1000 psig 70°F (21°C)


Component:	Concentration:	Accuracy:	Method:
Ethanol	260 ppm	+/- 0.002 or 2%	NDIR
Nitrogen	balance	BAC whichever is greater	

*NIST Standard Reference Material
 Cylinder No. CCI4290 / Job No. 09160202
 Certified 212.8 µmol/mol Ethanol in Nitrogen
 for ILMO Products Co., Jacksonville, IL

Store in dry area, away from sources of heat, ignition and direct sunlight. Do not allow storage area to exceed 52 °C (125 °F).

Jacob Matley
 Specialty Gas Lab Tech

09/06/16
 Date

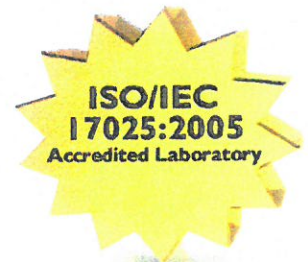

Distributed by:

CMI Inc.
 316 East Ninth Street
 Owensboro, KY 42303
 Phone 866-835-0690
www.alcoholtest.com

Moore

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST W. T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT
DATE 6-24-17 TIME 1748 HRS
INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE GOODYEAR AZ

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 009 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE W. T. Ryan #1100

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

06/24/2017

17:48:49

Standard Lot#: 23016100A1

Last Changed By: M. GUILBAULT #1144

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST

DOB: 06/24/1959

Sex: M Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:49:50
Diagnostic Test	Pass	17:50:25
Air Blank	0.000	17:50:52
0.100 Cal Check	0.100	17:51:13
Air Blank	0.000	17:51:43
Subject Test	0.000	17:51:58
Air Blank	ABT*	17:52:05
Air Blank	0.000	17:52:33

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

06/24/2017

17:52:50

Standard Lot#: 23016100A1

Last Changed By: M. GUILBAULT #1144

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST

DOB: 06/24/1959

Sex: M Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	17:53:34
Air Blank	0.000	17:54:02

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

06/24/2017

17:54:17

Standard Lot#: 23016100A1

Last Changed By: M. GUILBAULT #1144

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST

DOB: 06/24/1959

Sex: M Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	17:54:59
Air Blank	0.000	17:55:27

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

06/24/2017

17:55:46

Standard Lot#: 23016100A1

Last Changed By: M. GUILBAULT #1144

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 06/24/1959

Sex: M Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:56:46
Diagnostic Test	Pass	17:57:21
Air Blank	0.000	17:57:48
0.100 Cal Check	0.100	17:58:09
Air Blank	0.000	17:58:39
Subject Test	0.097*	18:01:46
Air Blank	0.000	18:02:18
Air Blank	0.000	18:02:45
Subject Test	XXX**	18:03:15
Air Blank	0.000	18:03:49

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

06/24/2017

18:04:13

Standard Lot#: 23016100A1

Last Changed By: M. GUILBAULT #1144

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	18:04:44
0.100 Cal Check	0.099	18:05:05
Air Blank	0.000	18:05:34

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT

DATE 6-24-17 TIME 1745

INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE GOODYEAR AZ

(✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0. 100 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan #1100

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

06/24/2017

17:45:58

Standard Lot#: 23016100A1

Last Changed By: M. GUILBAULT #1144

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	17:46:37
0.100 Cal Check	0.100	17:46:58
Air Blank	0.000	17:47:27

Preventative Maintenance
Performed -- Timer Reset