

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST W. T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT

DATE 6-24-17 TIME 1810 HRS

INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145TH AVE GOODYEAR AZ

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0.000 AC.
- 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 096 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE W. T. Ryan #1100

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

06/24/2017

18:10:22

Standard Lot#: 14816100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 06/24/1959

Sex: M Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:11:16
Diagnostic Test	Pass	18:11:51
Air Blank	0.000	18:12:19
0.100 Cal Check	0.096	18:12:39
Air Blank	0.000	18:13:09
Subject Test	0.000	18:13:31
Air Blank	ABT*	18:13:58
Air Blank	0.000	18:14:26

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-000312
Core Version: 8105.48
06/24/2017

18:14:55

Standard Lot#: 14816100A1
Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST
DOB: 06/24/1959
Sex: M Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	ABT*	18:15:31
Air Blank	0.000	18:15:59

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

06/24/2017

18:16:05

Standard Lot#: 14816100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 06/24/1959

Sex: M Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	18:16:45
Air Blank	0.000	18:17:14

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

06/24/2017

18:17:30

Standard Lot#: 14816100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 06/24/1959

Sex: M Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:18:24
Diagnostic Test	Pass	18:18:59
Air Blank	0.000	18:19:26
0.100 Cal Check	0.097	18:19:47
Air Blank	0.000	18:20:17
Subject Test	0.127*	18:23:25
Air Blank	0.000	18:23:57
Air Blank	0.000	18:24:24
Subject Test	XXX**	18:24:51
Air Blank	0.000	18:25:25

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

06/24/2017

18:26:18

Standard Lot#: 14816100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	18:26:50
0.100 Cal Check	0.096	18:27:11
Air Blank	0.000	18:27:41

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W. T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT
DATE 6-24-17 TIME 1757 HRS
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145TH AVE GOODYEAR AZ

- (V) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (V) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (V) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (V) 4. Air blank completed.
- (V) 5. Calibration check completed. Test results 0.097 AC.
- (V) 6. Air blank completed.
- (V) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W. T. Ryan 1100

INTOXILYZER 8000

Location: 11 N 145 AVE

Serial Number: 80-000312

Core Version: 8105.48

06/24/2017

17:57:50

Standard Lot#: 14816100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test g/210L Time

Air Blank 0.000 17:58:22
0.100 Cal Check 0.097 17:58:43
Air Blank 0.000 17:59:13

Preventative Maintenance
Performed -- Timer Reset