

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT

DATE 9-15-2017 TIME 1306 HRS

INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE, GOODYEAR AZ 85338

(✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0-100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0. 101 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan #1100

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

09/15/2017

13:06:12

Standard Lot#: 23016100A1

Last Changed By: M. GUILBAULT #1144

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	13:06:43
0.100 Cal Check	0.101	13:07:04
Air Blank	0.000	13:07:34

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST W. T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT

DATE 9-15-17 TIME 1309 HRS

INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE, GOODYEAR AZ 85338

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE W. T. Ryan #1100

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

09/15/2017

13:09:53

Standard Lot#: 23016100A1

Last Changed By: M. GUILBAULT #1144

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 09/15/1959

Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:11:21
Diagnostic Test	Pass	13:11:56
Air Blank	0.000	13:12:23
0.100 Cal Check	0.100	13:12:44
Air Blank	0.000	13:13:14
Subject Test	0.000	13:13:28
Air Blank	0.000	13:13:55
Wait	ABT*	
Air Blank	0.000	13:14:30

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

09/15/2017

13:15:00

Standard Lot#: 23016100A1

Last Changed By: M. GUILBAULT #1144

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 09/15/1959

Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	13:15:38
Air Blank	0.000	13:16:06

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

09/15/2017

13:16:39

Standard Lot#: 23016100A1

Last Changed By: M. GUILBAULT #1144

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST

DOB: 09/15/1959

Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	13:17:22
Air Blank	0.000	13:17:50

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
09/15/2017

13:18:33

Standard Lot#: 23016100A1
Last Changed By: M. GUILBAULT #1144

QAS: W. RYAN #1100
GOODYEAR PD

Operator: W. RYAN #1100
GOODYEAR PD

Subject: TEST,TEST
DOB: 09/15/1959
Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:19:28
Diagnostic Test	Pass	13:20:03
Air Blank	0.000	13:20:31
0.100 Cal Check	0.100	13:20:51
Air Blank	0.000	13:21:21
Subject Test	0.041*	13:24:28
Air Blank	0.000	13:25:00
Air Blank	0.000	13:25:27
Subject Test	XXX**	13:26:07
Air Blank	0.000	13:26:41

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

09/15/2017

13:27:27

Standard Lot#: 23016100A1

Last Changed By: M. GUILBAULT #1144

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	13:27:56
0.100 Cal Check	0.099	13:28:17
Air Blank	0.000	13:28:47

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GUELBAND AGENCY BOONVILLE PD
DATE 9/22/17 TIME 11:53
INTOXILYZER SERIAL # 89-001858 LOCATION CPA

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 100 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

DPS Form Exh G-2 (Rev 05-01)

CHECK PRINTER RE-CAL
[Signature]

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
09/22/2017

11:53:43

Standard Lot#: 23016100A1
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	11:54:12
0.100 Cal Check	0.100	11:54:33
Air Blank	0.000	11:55:03

Preventative Maintenance
Performed -- Timer Reset