

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

11/15/2017

14:47:47

-----  
Standard Lot#: 14816100A1

Last Changed By: W. RYAN #1100

-----  
QAS: W. RYAN #1100

GOODYEAR PD

-----  
Operator: W. RYAN #1100

GOODYEAR PD

-----  
Test                      g/210L                      Time  
-----  
Air Blank                      0.000                      14:48:17  
0.100 Cal Check                0.097                      14:48:38  
Air Blank                      0.000                      14:49:08

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. Ryan #1100 AGENCY GOODYEAR POLICE DEPT  
DATE 11-15-2017 TIME 1447  
INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145TH AVE, GOODYEAR AZ

- (S) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (GG) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(GG) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(GG) 4. Air blank completed.  
(GG) 5. Calibration check completed. Test results 0. 097 AC.  
(GG) 6. Air blank completed.  
(GG) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan #1100