

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. Ryan #1100 AGENCY GOODYEAR POLICE DEPT

DATE 2-12-18 TIME 1425 HRS

INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145TH AVE, GOODYEAR AZ 85338

(S) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (S) 2. Intoxilyzer 8000 display reads "~~PUSH BUTTON TO START~~" CONTACT GAS
- (S) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (S) 4. Air blank completed.
- (S) 5. Calibration check completed. Test results 0. 096 AC.
- (S) 6. Air blank completed.
- (S) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan 1100

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

02/12/2018

14:25:51

-----  
Standard Lot#: 14816100A1

Last Changed By: W. RYAN #1100

-----  
QAS: W. RYAN #1100

GOODYEAR PD

-----  
Operator: W. RYAN #1100

GOODYEAR PD

-----  
Test                      g/210L                      Time  
-----  
Air Blank                      0.000                      14:26:26  
0.100 Cal Check                0.096                      14:26:47  
Air Blank                      0.000                      14:27:17

Preventative Maintenance  
Performed -- Timer Reset