

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W. T. Ryan 1100 AGENCY GOODYEAR POLICE DEPT

DATE 2-12-18 TIME 1424 HRS

INTOXILYZER SERIAL # 90-09858 LOCATION 11 N 145TH AVE GOODYEAR AZ 85338

(S) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

(S) 2. Intoxilyzer 8000 display reads "~~PUSH BUTTON TO START~~" CONTACT QAS

(S) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

(S) 4. Air blank completed.

(S) 5. Calibration check completed. Test results 0. 100 AC.

(S) 6. Air blank completed.

(S) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W. T. Ryan 1100



EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST W. T. Ryan 1100 AGENCY GOODYEAR POLICE DEPT

DATE 2-20-18 TIME 2157

INTOXILYZER SERIAL # 80-001858 LOCATION 11N 145TH AVE GOODYEAR AZ 85338

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.

2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.

2. Error recognition logic system functioning.

Not a Successfully Completed Test Sequence printed

3. Proper sample recognition system.

Not a Successfully Completed Test Sequence printed

Deficient sample printed.

4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

TAKEN OUT OF SERVICE FOR DPS

DOWNLOAD, MAINT

SIGNATURE

W. T. Ryan #1100

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

02/20/2018

21:57:52

---

Standard Lot#: 23016100A1

Last Changed By: W. RYAN

#1100

---

QAS: W. RYAN

#1100

GOODYEAR PD

---

Operator: W. RYAN

#1100

GOODYEAR PD

---

Subject: TEST,TEST

DOB: 02/20/1959

Sex: M

Weight: 185

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	21:58:54
Diagnostic Test	Pass	21:59:29
Air Blank	0.000	21:59:56
0.100 Cal Check	0.100	22:00:17
Air Blank	0.000	22:00:46
Subject Test	0.000	22:01:05
Air Blank	0.000	22:01:32
Wait	ABT*	
Air Blank	0.000	22:02:11

\*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

02/20/2018

22:02:32

---

Standard Lot#: 23016100A1

Last Changed By: W. RYAN

#1100

---

QAS: W. RYAN

#1100

GOODYEAR PD

---

Operator: W. RYAN

#1100

GOODYEAR PD

---

Subject: TEST,TEST

DOB: 02/20/1959

Sex: M

Weight: 185

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	22:03:16
Air Blank	0.000	22:03:45

\*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

02/20/2018

22:04:16

---

Standard Lot#: 23016100A1

Last Changed By: W. RYAN

#1100

---

QAS: W. RYAN

#1100

GOODYEAR PD

---

Operator: W. RYAN

#1100

GOODYEAR PD

---

Subject: TEST,TEST

DOB: 02/20/1959

Sex: M

Weight: 185

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	22:04:57
Air Blank	0.000	22:05:26

\*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

02/20/2018

22:05:56

---

Standard Lot#: 23016100A1

Last Changed By: W. RYAN

#1100

---

QAS: W. RYAN

#1100

GOODYEAR PD

---

Operator: W. RYAN

#1100

GOODYEAR PD

---

Subject: TEST,TEST

DOB: 02/20/1959

Sex: M

Weight: 185

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	22:06:56
Diagnostic Test	Pass	22:07:31
Air Blank	0.000	22:07:58
0.100 Cal Check	0.100	22:08:19
Air Blank	0.000	22:08:49
Subject Test	0.159*	22:11:56
Air Blank	0.000	22:12:32
Air Blank	0.000	22:13:00
Subject Test	XXX**	22:13:35
Air Blank	0.000	22:14:08

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-001858  
Core Version: 8105.48  
02/20/2018

22:16:30

-----  
Standard Lot#: 23016100A1  
Last Changed By: W. RYAN #1100  
-----

QAS: W. RYAN #1100  
GOODYEAR PD  
-----

Operator: W. RYAN #1100  
GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	22:17:00
0.100 Cal Check	0.099	22:17:21
Air Blank	0.000	22:17:51

Preventative Maintenance  
Performed -- Timer Reset



EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W. T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT

DATE 2-20-18 TIME 2:55 HRS

INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE GOODYEAR AZ 85338

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.102 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W. T. Ryan 1100

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

02/20/2018

21:54:17

-----  
Standard Lot#: 23016100A1

Last Changed By: W. RYAN #1100  
-----

QAS: W. RYAN #1100

GOODYEAR PD  
-----

Operator: W. RYAN #1100

GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	21:55:00
0.100 Cal Check	0.102	21:55:21
Air Blank	0.000	21:55:51

Preventative Maintenance  
Performed -- Timer Reset



ARIZONA DEPARTMENT OF PUBLIC SAFETY  

---

INTOXILYZER MAINTENANCE REPORT

CR NO. 1800018580007

Page 1 of 1

AGENCY            Goodyear Police Department  
                      Goodyear, AZ 85338  
OFFICER           RYAN, #1100  
DATE                February 22, 2018



**INSTRUMENT**

80001858

**INSTRUMENT LOCATION**

Goodyear Police Department

**MAINTENANCE REQUESTED**

Repair and Preventative Maintenance

**MAINTENANCE PERFORMED**

***Parts Replaced:***

Power Supply

***Preventative Maintenance Performed:***

Inspection, Calibration, Verification

A handwritten signature in black ink, appearing to read "Joseph M Slowinski".

---

JOSEPH M SLOWINSKI, #6101, Electronics Technician  
Central Regional Crime Laboratory  
2102 W. Encanto Blvd., Phoenix, AZ 85009  
602-223-2159