

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR POLICE DEPT
DATE 4-13-06 TIME 1515
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD
QA SPECIALIST JEFF ROGERS
(print name)

1.

Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC. LOT 432401L

OR

Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

2.

3.

Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

4.

Air blank completed.

5.

Calibration check completed. Test results 0. 100 AC.

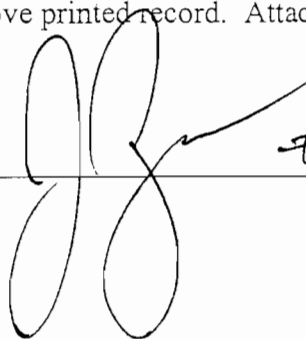
6.

Air blank completed.

7.

Remove printed record. Attach the record to the completed checklist.

SIGNATURE: _____

 #1123

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

04/13/2006

15:18:07

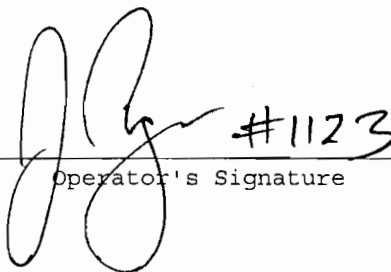
Standard Lot#: 432401I

Last Changed By: J. VALLANDINGHAM #981

QAS: J. ROGERS #1123

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	15:18:34
Cal Check	0.100	15:18:53
Air Blank	0.000	15:19:22

 #1123

Operator's Signature

EXHIBIT 3

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

AGENCY GOODYEAR P.D.
DATE 4-13-06 TIME 1520
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR P.D.
QA SPECIALIST JEFF ROGERS
(Print Name)

~~1~~ 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

LOT 432401L

~~1~~ 1. Clock time check.

~~2~~ 2. Date check.

OPERATIONAL TESTS

~~1~~ 1. Alcohol-free subject test result 0. 000 AC.

~~2~~ 2. ✓ Error recognition logic system functioning.

Not a Successfully Completed Test Sequence printed

~~3~~ 3. Proper sample recognition system.

Not a Successfully Completed Test Sequence printed

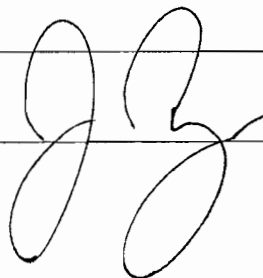
Deficient sample printed.

~~4~~ 4. Calibration standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES IL NO

COMMENTS:

SIGNATURE

 #1123

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

04/13/2006

15:30:07

Standard Lot#: 432401I

Last Changed By: J. VALLANDINGHAM #981

QAS: J. ROGERS #1123
GOODYEAR PD

Operator: J. ROGERS #1123
GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 200

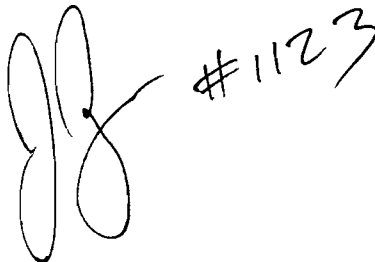
15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:30:59
Diagnostic Test	Pass	15:31:27
Air Blank	0.000	15:31:54
0.100 Cal Check	0.099	15:32:13
Air Blank	0.000	15:32:42
Subject Test	0.102*	15:35:51
Air Blank	0.000	15:36:23
Air Blank	0.000	15:36:50
Subject Test	XXX**	15:37:12
Air Blank	0.000	15:37:47

*Deficient Sample

**Invalid Sample

Begin new deprivation period

A handwritten signature, possibly 'J. Rogers', is written in black ink. To the right of the signature, the number '#1123' is handwritten in a similar style.

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

04/13/2006

15:28:36

Standard Lot#: 432401I

Last Changed By: J. VALLANDINGHAM #981

QAS: J. ROGERS

#1123

GOODYEAR PD

Operator: J. ROGERS

#1123

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	15:29:13
Air Blank	0.000	15:29:42

*RFI Detect

 #1123

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

04/13/2006

15:26:50

Standard Lot#: 432401I

Last Changed By: J. VALLANDINGHAM #981

QAS: J. ROGERS #1123
GOODYEAR PD

Operator: J. ROGERS #1123
GOODYEAR PD

Subject: TEST,TEST

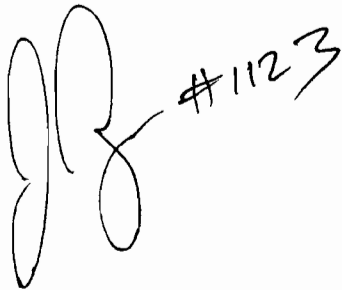
DOB: 01/01/0101

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	15:27:35
Air Blank	0.000	15:28:04

*Improper Sample

A handwritten signature, possibly 'JR', followed by the number '#1123'.

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

04/13/2006

15:21:29

Standard Lot#: 432401I

Last Changed By: J. VALLANDINGHAM #981

QAS: J. ROGERS

#1123

GOODYEAR PD

Operator: J. ROGERS

#1123

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:22:55
Diagnostic Test	Pass	15:23:23
Air Blank	0.000	15:23:50
0.100 Cal Check	0.101	15:24:09
Air Blank	0.000	15:24:38
Subject Test	0.000	15:25:02
Air Blank	0.000	15:25:29
Five Minute Wait	ABT*	
Air Blank	0.000	15:26:07

*Sequence Aborted

 #1123

Not a Successfully Completed Test Sequence

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR P.D.
DATE 4-13-06 TIME 1536
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR P.D.
QA SPECIALIST JEFF ROGERS
(print name)

~~1~~ 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC. LOT 432401L

OR

Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

~~2~~ 2.
~~3~~ 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

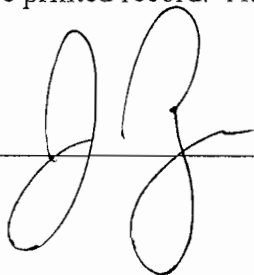
~~4~~ 4. Air blank completed.

~~5~~ 5. Calibration check completed. Test results 0. 099 AC.

~~6~~ 6. Air blank completed.

~~7~~ 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: _____

 #1123

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

04/13/2006

15:39:06

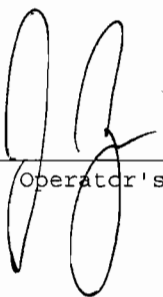
Standard Lot#: 432401I

Last Changed By: J. VALLANDINGHAM #981

QAS: J. ROGERS #1123

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	15:39:33
Cal Check	0.099	15:39:52
Air Blank	0.000	15:40:22

 #1123

Operator's Signature