

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

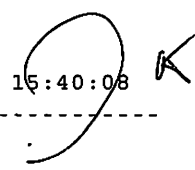
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. Gouilbault 1144 AGENCY Goodyear PD  
DATE 6/5/18 TIME 1540  
INTOXILYZER SERIAL # 80-000312 LOCATION 6PD

- ( ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- ( ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( ) 4. Air blank completed.
- ( ) 5. Calibration check completed. Test results 0. 096 AC.
- ( ) 6. Air blank completed.
- ( ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE AA 1144

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-000312  
Core Version: 8105.48  
06/05/2018

15:40:08 

-----  
Standard Lot#: 14816100A1  
Last Changed By: W. RYAN #1100  
-----

QAS: M. GUILBAULT #1144  
GOODYEAR PD  
-----

Operator: M. GUILBAULT #1144  
GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	15:40:38
0.100 Cal Check	0.096	15:40:58
Air Blank	0.000	15:41:28

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. Gui'lhavlt 1144 AGENCY Goodyear PD  
DATE 6/5/18 TIME 1543  
INTOXILYZER SERIAL # 80-000312 LOCATION GPA

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.  
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.  
 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 094 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE MA 1144

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-000312  
Core Version: 8105.48  
06/05/2018

15:43:50

---

Standard Lot#: 14816100A1  
Last Changed By: W. RYAN #1100

---

QAS: M. GUILBAULT #1144  
GOODYEAR PD

---

Operator: M. GUILBAULT #1144  
GOODYEAR PD

---

Subject: TEST,TEST  
DOB: 01/01/1990  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	15:44:26
Air Blank	RFI**	15:44:33

\*Improper Sample  
\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-000312  
Core Version: 8105.48  
06/05/2018

15:44:36 *N*

---

Standard Lot#: 14816100A1  
Last Changed By: W. RYAN #1100

---

QAS: M. GUILBAULT #1144  
GOODYEAR PD

---

Operator: M. GUILBAULT #1144  
GOODYEAR PD

---

Subject: TEST,TEST  
DOB: 01/01/1990  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	15:45:28
Diagnostic Test	Pass	15:46:03
Air Blank	0.000	15:46:30
0.100 Cal Check	0.095	15:46:51
Air Blank	0.000	15:47:21
Subject Test	0.000	15:47:36
Air Blank	ABT*	15:47:42
Air Blank	0.000	15:48:10

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-000312  
Core Version: 8105.48  
06/05/2018

15:48:30

---

Standard Lot#: 14816100A1  
Last Changed By: W. RYAN #1100

---

QAS: M. GUILBAULT #1144  
GOODYEAR PD

---

Operator: M. GUILBAULT #1144  
GOODYEAR PD

---

Subject: TEST,TEST  
DOB: 01/01/1990  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	15:49:18
Diagnostic Test	Pass	15:49:53
Air Blank	0.000	15:50:20
0.100 Cal Check	0.095	15:50:41
Air Blank	0.000	15:51:11
Subject Test	0.143*	15:54:19
Air Blank	0.000	15:54:51
Air Blank	RFI**	15:55:05
Air Blank	0.000	15:55:33

\*Deficient Sample  
\*\*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-000312  
Core Version: 8105.48  
06/05/2018

15:55:36



---

Standard Lot#: 14816100A1  
Last Changed By: W. RYAN #1100

---

QAS: M. GUILBAULT #1144  
GOODYEAR PD

---

Operator: M. GUILBAULT #1144  
GOODYEAR PD

---

Subject: TEST,TEST  
DOB: 01/01/1990  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	15:56:22
Diagnostic Test	Pass	15:56:57
Air Blank	0.000	15:57:24
0.100 Cal Check	0.095	15:57:45
Air Blank	0.000	15:58:15
Subject Test	XXX*	15:58:44
Air Blank	0.000	15:59:18

\*Invalid Sample  
Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. Guilbault AGENCY Goodyear PD  
DATE 6/5/18 TIME 1559  
INTOXILYZER SERIAL # 80-000312 LOCATION 6PD

(\*) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (\*) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (\*) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (\*) 4. Air blank completed.
- (\*) 5. Calibration check completed. Test results 0.094 AC.
- (\*) 6. Air blank completed.
- (\*) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE AG 1144



INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-000312  
Core Version: 8105.48  
06/05/2018

15:59:32

-----  
Standard Lot#: 14816100A1  
Last Changed By: W. RYAN #1100  
-----

QAS: M. GUILBAULT #1144  
GOODYEAR PD  
-----

Operator: M. GUILBAULT #1144  
GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	16:00:02
0.100 Cal Check	0.094	16:00:22
Air Blank	0.000	16:00:53

Preventative Maintenance  
Performed -- Timer Reset