

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR POLICE DEPARTMENT
DATE 8/10/2004 TIME 0929
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD
QA SPECIALIST Will
(print name)

- (u) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard 100 AC.
- OR
- (u) 2. Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (u) 3. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (u) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (u) 4. Air blank completed.
- (u) 5. Calibration check completed. Test results 0. 098 AC.
- (u) 6. Air blank completed.
- (u) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: _____

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

08/10/2004

09:29:55

Standard Lot#: 325402G

Last Changed By: W. WILL #974

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	09:30:22
Cal Check	0.098	09:30:41
Air Blank	0.000	09:31:10

Operator's Signature

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR P.D.
DATE 8-10-2004 TIME 14:55
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR P.D.
QA SPECIALIST J. VALLANDINGHAM # 981
(print name)

- (☒) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC.
- OR
- Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (☒) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (☒) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (☒) 4. Air blank completed.
- (☒) 5. Calibration check completed. Test results 0. 098 AC.
- (☒) 6. Air blank completed.
- (☒) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: _____

J. Vallandingham #981 QAS

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

08/10/2004

14:55:11

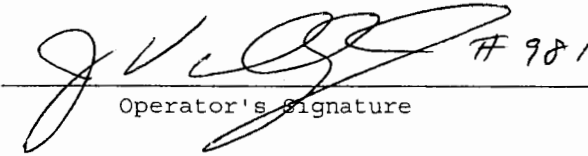
Standard Lot#: 325402G

Last Changed By: W. WILL #974

QAS: J. VALLANDINGHAM #981

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	14:55:38
Cal Check	0.098	14:55:57
Air Blank	0.000	14:56:27

 #981 QAS

Operator's Signature