

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR PD
DATE 8-08-05 TIME 0026
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD BOOKING AREA
QA SPECIALIST J. VALLANDINGHAM #981
(print name)

(☒ 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC.

OR

Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

(☒ 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

(☒ 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

(☒ 4. Air blank completed.

(☒ 5. Calibration check completed. Test results 0. 097 AC.

(☒ 6. Air blank completed.

(☒ 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: _____



INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

08/08/2005

00:26:14


Standard Lot#: 422601G

Last Changed By: W. WILL #974

QAS: J. VALLANDINGHAM #981

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	00:26:42
Cal Check	0.097	00:27:01
Air Blank	0.000	00:27:30

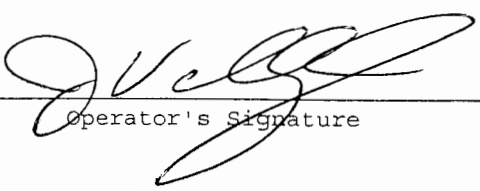

Operator's Signature

EXHIBIT 3

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

AGENCY GOODYEAR PDDATE 8-08-05 TIME 0033INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD BOOKING AREAQA SPECIALIST J. VALLANDINGHAM #981
(Print Name)☒ 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

☒ 1. Clock time check.☒ 2. Date check. _____

OPERATIONAL TESTS

☒ 1. Alcohol-free subject test result 0. 000 AC.☒ 2. Error recognition logic system functioning.

Not a Successfully Completed Test Sequence printed

☒ 3. Proper sample recognition system.

Not a Successfully Completed Test Sequence printed

Deficient sample printed.

☒ 4. Calibration standard 0. 100 AC. Result 0. 096 AC.Instrument is operating properly and accurately. YES ☒ NO ☐

COMMENTS: _____

SIGNATURE 

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

08/08/2005

00:33:11

Standard Lot#: 422601G

Last Changed By: W. WILL

#974

QAS: J. VALLANDINGHAM

#981

GOODYEAR PD

Operator: J. VALLANDINGHAM

#981

GOODYEAR PD

Subject: SQAP,TEST

DOB: 11/11/1111

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	00:34:35
Diagnostic Test	Pass	00:35:04
Air Blank	0.000	00:35:31
0.100 Cal Check	0.096	00:35:50
Air Blank	0.000	00:36:19
Subject Test	0.000	00:36:47
Air Blank	0.000	00:37:15
Five Minute Wait	ABT*	
Air Blank	0.000	00:37:52

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: GOODYEAR POLICE DEPT
Serial Number : 80-000312
08/08/2005 00:40:37

Standard Lot#: 422601G
Last Changed By: W. WILL #974

QAS: J. VALLANDINGHAM #981
GOODYEAR PD

Operator: J. VALLANDINGHAM #981
GOODYEAR PD

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	00:41:40
Air Blank	0.000	00:42:09

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

08/08/2005

00:42:39

Standard Lot#: 422601G

Last Changed By: W. WILL

#974

QAS: J. VALLANDINGHAM

#981

GOODYEAR PD

Operator: J. VALLANDINGHAM

#981

GOODYEAR PD

Subject: SQAP, TEST

DOB: 11/11/1111

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	00:43:44
Air Blank	0.000	00:44:12

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

08/08/2005

00:44:38

Standard Lot#: 422601G

Last Changed By: W. WILL

#974

QAS: J. VALLANDINGHAM

#981

GOODYEAR PD

Operator: J. VALLANDINGHAM

#981

GOODYEAR PD

Subject: SQAP, TEST

DOB: 11/11/1111

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	00:45:44
Diagnostic Test	Pass	00:46:12
Air Blank	0.000	00:46:39
0.100 Cal Check	0.097	00:46:58
Air Blank	0.000	00:47:27
Subject Test	0.162*	00:50:36
Air Blank	0.000	00:51:07
Air Blank	0.000	00:51:35
Subject Test	XXX**	00:51:52
Air Blank	0.000	00:52:26

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR PD
DATE 8-08-05 TIME 0053
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD BOOKING AREA
QA SPECIALIST J. VALLANDINGHAM #981
(print name)

(☒) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC.

OR

Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

(☒) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

(☒) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

(☒) 4. Air blank completed.

(☒) 5. Calibration check completed. Test results 0. 096 AC.

(☒) 6. Air blank completed.

(☒) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: _____



INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

08/08/2005

00:53:21


Standard Lot#: 422601G

Last Changed By: W. WILL #974

QAS: J. VALLANDINGHAM #981

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	00:53:48
Cal Check	0.096	00:54:07
Air Blank	0.000	00:54:36

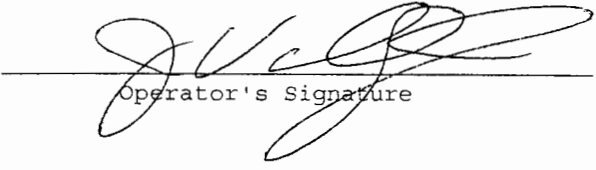

Operator's Signature