

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR POLICE DEPARTMENT  
DATE 12/24/04 TIME 0045  
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD  
QA SPECIALIST Will  
( print name)

- (u) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard 100 AC.  

OR

Pour a standard alcohol solution of known value, \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (u) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (u) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (u) 4. Air blank completed.
- (u) 5. Calibration check completed. Test results 0. 098 AC.
- (u) 6. Air blank completed.
- (u) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: \_\_\_\_\_



INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

12/24/2004

00:45:50

-----  
Standard Lot#: 325402G

Last Changed By: W. WILL #974

-----  
QAS: W. WILL #974

GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	00:46:18
Cal Check	0.098	00:46:37
Air Blank	0.000	00:47:06

\_\_\_\_\_  
Operator's Signature