

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR PD
DATE 12-18-2005 TIME 0021
INTOXILYZER SERIAL NO. 20-000312 LOCATION GOODYEAR PD BOOKING AREA
QA SPECIALIST J. VALLANDINGHAM #981
(print name)

- (☒) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC.
- OR**
- (☒) 2. Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (☒) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (☒) 4. Air blank completed.
- (☒) 5. Calibration check completed. Test results 0.100 AC.
- (☒) 6. Air blank completed.
- (☒) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: _____



INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

12/18/2005

00:21:51

Standard Lot#: 431001G

Last Changed By: W. WILL #974

QAS: J. VALLANDINGHAM #981

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	00:22:18
Cal Check	0.100	00:22:37
Air Blank	0.000	00:23:07

 #981

Operator's Signature

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR PD
DATE 12-25-2005 TIME 1117
INTOXILYZER SERIAL NO 80-000312 LOCATION GOODYEAR PD - MAIN STATION
QA SPECIALIST J. VALLANDINGHAM #981
(print name)

- (☒) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard 0.100 AC.
- OR
- (☒) 2. Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (☒) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (☒) 4. Air blank completed.
- (☒) 5. Calibration check completed. Test results 0.099 AC.
- (☒) 6. Air blank completed.
- (☒) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: _____



INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

12/25/2005

11:17:12

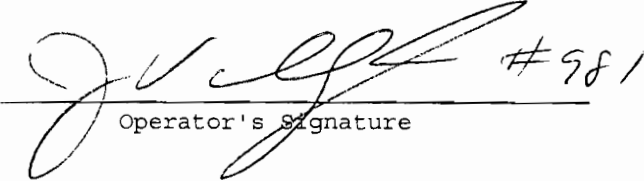
Standard Lot#: 431001G

Last Changed By: W. WILL #974

QAS: J. VALLANDINGHAM #981

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	11:17:40
Cal Check	0.099	11:17:58
Air Blank	0.000	11:18:28

 #981

Operator's Signature