

B

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GULBAULT AGENCY GOODYEAR PD
DATE 12/3/12 TIME 1550
INTOXILYZER SERIAL # 80-000312 LOCATION 111 S LITCHFIELD RD

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 102 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1144

INTOXILYZER 8000

Location: 1111 S LICHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

12/03/2012

15:50:45

Standard Lot#: 34011100A1

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	15:51:15
0.100 Cal Check	0.102	15:51:36
Air Blank	0.000	15:52:06

Preventative Maintenance
Performed -- Timer Reset

20

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. G. ELMAULT AGENCY GOODYEAR PD
DATE 12/3/12 TIME 1553
INTOXILYZER SERIAL # 80-000312 LOCATION 111 S LITCHFIELD RD

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 101 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: 1111 S LICHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

12/03/2012

15:53:59

Standard Lot#: 13712100A2

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	15:54:28
0.100 Cal Check	0.101	15:54:49
Air Blank	0.000	15:55:19

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. GILBERT AGENCY GOODYEAR PD
DATE 12/3/2012 TIME 1555
INTOXILYZER SERIAL # 80-000312 LOCATION 111 S LITCHFIELD A

☒ 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- ☒ 1. Clock time check.
☒ 2. Date check.

OPERATIONAL TESTS

- ☒ 1. Alcohol-free subject test result 0. 000 AC.
☒ 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
☒ 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES ☒ NO ☐

COMMENTS

SIGNATURE W. 1144

INTOXILYZER 8000
Location: 1111 S LICHFIELD RD
Serial Number: 80-000312
Core Version: 8105.48
12/03/2012

15:55:28

Standard Lot#: 13712100A2
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/1990

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:56:15
Diagnostic Test	Pass	15:56:50
Air Blank	0.000	15:57:17
0.100 Cal Check	0.101	15:57:38
Air Blank	0.000	15:58:08
Subject Test	0.000	15:58:29
Air Blank	IPS*	15:58:36
Air Blank	RFI**	15:58:45

*Improper Sample

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LICHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

12/03/2012

15:58:53

Standard Lot#: 13712100A2

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/1990

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:59:38
Diagnostic Test	Pass	16:00:13
Air Blank	0.000	16:00:41
Reference	ABT*	16:00:46
Air Blank	0.000	16:01:14

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 1111 S LICHFIELD RD
Serial Number: 80-000312
Core Version: 8105.48
12/03/2012

16:01:22

Standard Lot#: 13712100A2
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Subject: TEST,TEST
DOB: 01/01/1990
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	16:02:13
Diagnostic Test	Pass	16:02:48
Air Blank	0.000	16:03:15
0.100 Cal Check	0.100	16:03:36
Air Blank	0.000	16:04:06
Subject Test	0.017*	16:07:13
Air Blank	0.000	16:07:45
Air Blank	0.000	16:08:12
Subject Test	XXX**	16:08:31
Air Blank	0.000	16:09:05

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

(12)

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GULHAULT AGENCY GOODYEAR PD

DATE 12/3/2012 TIME 1611

INTOXILYZER SERIAL # 80-000312 LOCATION 1111 S LITCHFIELD RD

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(X) 4. Air blank completed.
(X) 5. Calibration check completed. Test results 0. 100 AC.
(X) 6. Air blank completed.
(X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000
Location: 1111 S LICHFIELD RD
Serial Number: 80-000312
Core Version: 8105.48
12/03/2012

16:11:11

Standard Lot#: 13712100A2
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	16:11:40
0.100 Cal Check	0.100	16:12:01
Air Blank	0.000	16:12:31

Preventative Maintenance
Performed -- Timer Reset



7 Eastgate Dr. • P.O. Box 790 • Jacksonville, IL 62651-0790
217-245-2183 • Fax: 217-243-7634 • www.ilmoproducts.com

Certificate of Analysis

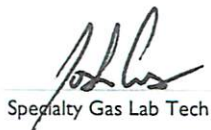
Certificate ID: 3341
Part #: BAC105L100T
Cylinder Size: 105L
Lot Number: 13712100A2
Expiration: 7/1/2014

0.100 BAC (For use with breath alcohol testing instruments)

Contents: 105 Liters @ 1000 psig 70°F (21°C)

Component:	Concentration:	Accuracy:	Method:
Ethanol	260.5 ppm	+/- 0.002 or 2%	NDIR
Nitrogen	Balance	BAC whichever is greater	

*NIST Standard Reference Material
Certification of NTRM Batch No. 091602
Nominal 210 $\mu\text{mol/mol}$ Ethanol in Nitrogen
for ILMO Products Co., Jacksonville, IL


Specialty Gas Lab Tech

5-25-12
Date



EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GUTENOWIT 11K4 AGENCY GOOD YEAR PD
DATE 12/25/12 TIME 2124
INTOXILYZER SERIAL # 50-000312 LOCATION 1111 S LITCHFIELD RD

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 101 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: 1111 S LICHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

12/27/2012

21:24:39

Standard Lot#: 13712100A2

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	21:25:15
0.100 Cal Check	0.101	21:25:35
Air Blank	0.000	21:26:05

Preventative Maintenance
Performed -- Timer Reset