

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR POLICE DEPARTMENT
DATE 2/18/2005 TIME 14:47
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD
QA SPECIALIST Will
(print name)

- (u) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard 100 AC.
- OR
- Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (u) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (u) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (u) 4. Air blank completed.
- (u) 5. Calibration check completed. Test results 0. 098 AC.
- (u) 6. Air blank completed.
- (u) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: 

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

02/18/2005

14:47:51

Standard Lot#: 422601G

Last Changed By: W. WILL #974

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	14:48:18
Cal Check	0.098	14:48:37
Air Blank	0.000	14:49:06

Operator's Signature

EXHIBIT 3
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

AGENCY GOODYEAR POLICE DEPARTMENT
DATE 2/18/2005 TIME 1447
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD
QA SPECIALIST WILL
(Print Name)

(u) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

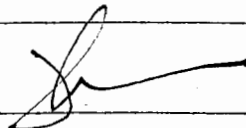
(u) 1. Clock time check.
(u) 2. Date check: _____

OPERATIONAL TESTS

(u) 1. Alcohol-free subject test result 0. 000 AC.
(u) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(u) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(u) 4. Calibration standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES X NO _____

COMMENTS: _____

SIGNATURE 

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

02/18/2005

14:49:35

Standard Lot#: 422601G

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:50:27
Diagnostic Test	Pass	14:50:54
Air Blank	0.000	14:51:21
0.100 Cal Check	0.099	14:51:40
Air Blank	0.000	14:52:09
Subject Test	0.000	14:52:29
Air Blank	0.000	14:52:56
Five Minute Wait	ABT*	
Air Blank	0.000	14:53:31

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

02/18/2005

14:53:36

Standard Lot#: 422601G

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	14:54:33
Air Blank	0.000	14:55:01

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

02/18/2005

14:55:06

Standard Lot#: 422601G

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	14:55:35
Air Blank	0.000	14:56:03

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

02/18/2005

14:56:08

Standard Lot#: 422601G

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:56:57
Diagnostic Test	Pass	14:57:25
Air Blank	0.000	14:57:52
0.100 Cal Check	0.098	14:58:11
Air Blank	0.000	14:58:40
Subject Test	0.302*	15:01:49
Air Blank	0.000	15:02:21
Air Blank	0.000	15:02:48
Subject Test	XXX**	15:03:08
Air Blank	0.000	15:03:40

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

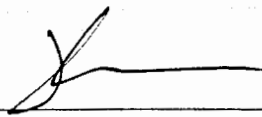
EXHIBIT 2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR POLICE DEPARTMENT
DATE 2/18/2005 TIME 1503
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD
QA SPECIALIST Will
(print name)

- (u) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC.
OR
Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (u) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (u) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (u) 4. Air blank completed.
- (-) 5. Calibration check completed. Test results 0. 098 AC.
- (u) 6. Air blank completed.
- (u) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: 

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

02/18/2005

15:03:59

Standard Lot#: 422601G

Last Changed By: W. WILL #974

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	15:04:26
Cal Check	0.098	15:04:45
Air Blank	0.000	15:05:15

Operator's Signature