

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR POLICE DEPARTMENT
DATE 1/23/2004 TIME 1155
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD
QA SPECIALIST Will
(print name)

- () 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard ~~0.000~~ 100 AC.
- OR
- Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (u) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (u) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (u) 4. Air blank completed.
- (u) 5. Calibration check completed. Test results 0. 0.99 AC.
- (u) 6. Air blank completed.
- (u) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: _____

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

01/23/2004

11:55:20

Standard Lot#: 321802I

Last Changed By: W. WILL #974

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	11:55:47
Cal Check	0.099	11:56:05
Air Blank	0.000	11:56:34



Operator's Signature



Date: March 2, 2004
To: Intoxilyzer Record Book
From: Officer Jeff Vallandingham #981
Ref: Information

On January 23, 2004 at approximately 1813 hours, I performed a calibration check, a SQAP, and a final calibration check (exhibits 2, 3 , and 2) of the Intoxilyzer 8000 serial number 80-000312. During the process I was in the QAS menu and accidentally pressed yes as changing the standard lot #321802I when in fact I never actually changed the standard lot.

A handwritten signature in black ink, appearing to read "Jeff Vallandingham". The signature is stylized and cursive.

Jeff Vallandingham

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR P.D.
DATE 1-23-2004 TIME 1813
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR P.D.
QA SPECIALIST J. VALLANDINGHAM
(print name)

- (✓) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC.
OR
Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ±0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 098 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: _____

J. Vallandigham

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

01/23/2004

18:13:54


Standard Lot#: 321802I

Last Changed By: W. WILL #974

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	18:14:21
Cal Check	0.098	18:14:39
Air Blank	0.000	18:15:08

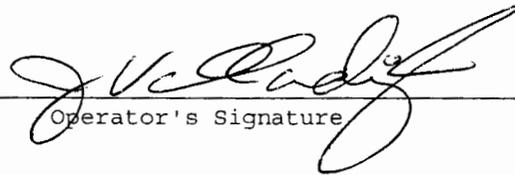

Operator's Signature

EXHIBIT 3

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

AGENCY GOODYEAR P.D.
DATE 1-23-2004 TIME 1827
INTOXILYZER SERIAL NO. 80-00312 LOCATION GOODYEAR P.D.
QA SPECIALIST J. VALLANDINGHAM
(Print Name)

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

() 1. Clock time check.

() 2. Date check.

OPERATIONAL TESTS

() 1. Alcohol-free subject test result 0. 000 AC.

() 2. Error recognition logic system functioning.

Not a Successfully Completed Test Sequence printed

() 3. Proper sample recognition system.


Not a Successfully Completed Test Sequence printed

Deficient sample printed.

() 4. Calibration standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS: _____

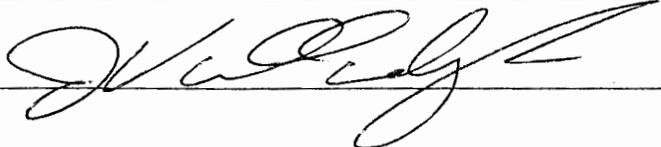SIGNATURE 

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

01/23/2004

18:27:25

Standard Lot#: 321802I

Last Changed By: W. WILL

#974

QAS: J. VALLANDINGHAM

#981

GOODYEAR PD

Operator: J. VALLANDINGHAM

#981

GOODYEAR PD

Subject: SQAP,TEST

DOB: 11/11/1111

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:28:22
Diagnostic Test	Pass	18:28:49
Air Blank	0.000	18:29:15
0.100 Cal Check	0.098	18:29:34
Air Blank	0.000	18:30:03
Subject Test	0.000	18:30:20
Air Blank	0.000	18:30:46
Five Minute Wait	ABT*	
Air Blank	0.000	18:31:23

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

01/23/2004

18:31:57

Standard Lot#: 321802I

Last Changed By: W. WILL

#974

QAS: J. VALLANDINGHAM

#981

GOODYEAR PD

Operator: J. VALLANDINGHAM

#981

GOODYEAR PD

Subject: SQAP,TEST

DOB: 11/11/1111

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	18:32:40
Air Blank	0.000	18:33:08

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

01/23/2004

18:34:36

Standard Lot#: 321802I

Last Changed By: W. WILL

#974

QAS: J. VALLANDINGHAM

#981

GOODYEAR PD

Operator: J. VALLANDINGHAM

#981

GOODYEAR PD

Subject: SQAP,TEST,N

DOB: 11/11/1111

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	18:35:18
Air Blank	0.000	18:35:46

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: GOODYEAR POLICE DEPT
Serial Number : 80-000312
01/23/2004 18:36:51

Standard Lot#: 321802I
Last Changed By: W. WILL #974

QAS: J. VALLANDINGHAM #981
GOODYEAR PD

Operator: J. VALLANDINGHAM #981
GOODYEAR PD

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:37:58
Diagnostic Test	Pass	18:38:25
Air Blank	0.000	18:38:52
0.100 Cal Check	0.098	18:39:10
Air Blank	0.000	18:39:39
Subject Test	0.010*	18:42:47
Air Blank	0.000	18:43:18
Air Blank	0.000	18:43:45
Subject Test	XXX**	18:44:25
Air Blank	0.000	18:44:59

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES


STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY Goodyear P.D.
DATE 1-23-2004 TIME 1847
INTOXILYZER SERIAL NO. 80-000 312 LOCATION Goodyear P.D.
QA SPECIALIST J. VALLANDINGHAM
(print name)

- () 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC.
- OR
- Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ±0.2°C
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 097 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: _____

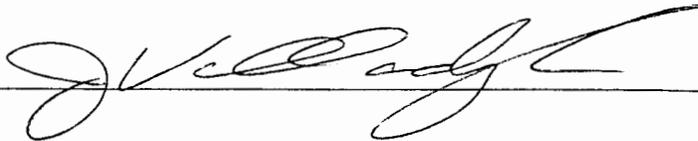

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

01/23/2004

18:47:16


Standard Lot#: 321802I

Last Changed By: W. WILL #974

QAS: J. VALLANDINGHAM #981

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	18:47:43
Cal Check	0.097	18:48:01
Air Blank	0.000	18:48:30

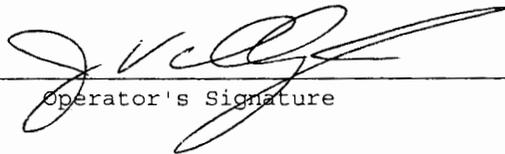

Operator's Signature