

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. ROGERS AGENCY GOODYEAR P.D.
DATE 1-21-07 TIME 1520
INTOXILYZER SERIAL # 80-000312 LOCATION GOODYEAR P.D.

- ☒ 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- ☒ 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ☒ 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ☒ 4. Air blank completed.
- ☒ 5. Calibration check completed. Test results 0. 100 AC.
- ☒ 6. Air blank completed.
- ☒ 7. Remove printed record. Attach the record to the completed checklist.
- LOT 618601I
EXP 7-7-08

SIGNATURE [Signature] # 1123

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

01/21/2007

15:20:17

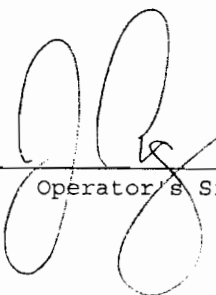
Standard Lot#: 618601I

Last Changed By: J. ROGERS #1123

QAS: J. ROGERS #1123

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	15:20:44
Cal Check	0.100	15:21:03
Air Blank	0.000	15:21:33

 #1123

Operator's Signature