

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GUSEBAULT AGENCY GOODYEAR PD
DATE 1/7/2011 TIME 1059
INTOXILYZER SERIAL # 80-000312 LOCATION 10701 S ESTRELLA PKWY

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 103 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000
Location: 10701 S ESTRELLA
Serial Number: 80-000312
Core Version: 8105.48
01/07/2011

10:59:32

Standard Lot#: 767153
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	11:00:22
0.100 Cal Check	0.103	11:00:43
Air Blank	0.000	11:01:12

Preventative Maintenance
Performed -- Timer Reset


1144

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. GUYERHULT AGENCY GOODYEAR PD
DATE 1/7/2011 TIME 1101
INTOXILYZER SERIAL # 80-000212 LOCATION 10701 SESTRELLA PKWY

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 102 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE [Signature] 1144

INTOXILYZER 8000
Location: 10701 S ESTRELLA
Serial Number: 80-000312
Core Version: 8105.48
01/07/2011

11:01:31

Standard Lot#: 767153
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Subject: TEST,TEST
DOB: 01/01/1990
Sex: M Weight: 120

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:02:27
Diagnostic Test	Pass	11:03:02
Air Blank	0.000	11:03:29
0.100 Cal Check	0.103	11:03:50
Air Blank	0.000	11:04:19
Subject Test	0.000	11:04:31
Air Blank	0.000	11:04:58
Wait	ABT*	
Air Blank	0.000	11:05:33

*Sequence Aborted

1144

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 10701 S ESTRELLA
Serial Number: 80-000312
Core Version: 8105.48
01/07/2011

11:05:37

Standard Lot#: 767153
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Subject: TEST, TEST
DOB: 01/01/1990
Sex: M Weight: 10

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	11:06:10
Air Blank	RFI**	11:06:21

*Improper Sample
**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 10701 S ESTRELLA
Serial Number: 80-000312
Core Version: 8105.48
01/07/2011

11:06:26

Standard Lot#: 767153
Last Changed By: M. GUILBAULT #1144
QAS: M. GUILBAULT #1144
GOODYEAR PD
Operator: M. GUILBAULT #1144
GOODYEAR PD

Subject: TEST, TEST
DOB: 01/01/1990
Sex: M Weight: 120

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:07:15
Diagnostic Test	Pass	11:07:50
Air Blank	0.000	11:08:17
0.100 Cal Check	0.102	11:08:38
Air Blank	0.000	11:09:08
Subject Test	0.437*	11:12:15
Air Blank	0.000	11:12:48
Air Blank	0.000	11:13:15
Subject Test	XXX**	11:13:39
Air Blank	0.000	11:14:13

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. G. BAULT AGENCY GOODYEAR PD
DATE 1/7/2011 TIME 1115
INTOXILYZER SERIAL # 80-000712 LOCATION 10701 S ESTRELLA PKWY

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 102 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1144

INTOXILYZER 8000
Location: 10701 S ESTRELLA
Serial Number: 80-000312
Core Version: 8105.48
01/07/2011

11:15:03

Standard Lot#: 767153
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	11:15:32
0.100 Cal Check	0.102	11:15:53
Air Blank	0.000	11:16:23

Preventative Maintenance
Performed -- Timer Reset


EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GULBAULT AGENCY GOODYEAR PD
DATE 1/7/2011 TIME 1323
INTOXILYZER SERIAL # 80-000312 LOCATION 10701 SESTRELLA Pkwy

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 101 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

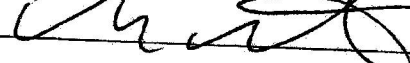SIGNATURE 

INTOXILYZER 8000
Location: 10701 S ESTRELLA
Serial Number: 80-000312
Core Version: 8105.48
01/07/2011

13:23:09

Standard Lot#: 16610100A1
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	13:23:52
0.100 Cal Check	0.101	13:24:13
Air Blank	0.000	13:24:43

Preventative Maintenance
Performed -- Timer Reset

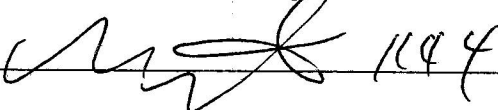
M. Guilbault 1144

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GUSBAUER AGENCY GOODLEIGH PD
DATE 11/31/2011 TIME 0852
INTOXILYZER SERIAL # 80-000312 LOCATION 10701 S ESTRELA Pkwy

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 101 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  1144

INTOXILYZER 8000
Location: 10701 S ESTRELLA
Serial Number: 80-000312
Core Version: 8105.48
01/31/2011

08:51:16

Standard Lot#: 16610100A1
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	08:52:10
0.100 Cal Check	0.101	08:52:31
Air Blank	0.000	08:53:01

Preventative Maintenance
Performed -- Timer Reset