

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GUILBAULT AGENCY GOODYEAR PD
DATE 11/24/2012 TIME 1934
INTOXILYZER SERIAL # 80-000312 LOCATION 10701 S ESTRELLA PKWY

- ☒ 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- ☒ 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ☒ 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ☒ 4. Air blank completed.
- ☒ 5. Calibration check completed. Test results 0. 102 AC.
- ☒ 6. Air blank completed.
- ☒ 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: 10701 S ESTRELLA

Serial Number: 80-000312

Core Version: 8105.48

01/24/2012

19:34:28

Standard Lot#: 06011100A1

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	19:35:13
0.100 Cal Check	0.102	19:35:33
Air Blank	0.000	19:36:03

Preventative Maintenance
Performed -- Timer Reset