

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN 1100 AGENCY GOODYEAR POLICE DEPT
DATE 1-9-2014 TIME 1904
INTOXILYZER SERIAL # 80-000312 LOCATION 1111 S LITCHFIELD RD GOODYEAR AZ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.10 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

W.T. Ryan 1100

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

01/09/2014

19:04:11

Standard Lot#: 13712100A2

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	19:04:48
0.100 Cal Check	0.100	19:05:09
Air Blank	0.000	19:05:39

Preventative Maintenance

Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. Gribbault AGENCY Good Year PD
DATE 1/29/2014 TIME 2334
INTOXILYZER SERIAL # P0-000312 LOCATION 1111 S LITCHFIELD RD.

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 100 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

01/29/2014

23:34:48

Standard Lot#: 13712100A2

Last Changed By: W. RYAN #1100

QAS: M. GUILBAULT #1144

GOODYEAR P)

Operator: M. GUILBAULT #1144

GOODYEAR P)

Test	g/210L	Time
✓ Air Blank	0.000	23:35:19
0.100 Cal Check	0.100	23:35:40
Air Blank	0.000	23:36:10

Preventative Maintenance

Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. GUTENBERG AGENCY GOODYEAR PD
DATE 1/29/14 TIME 2336
INTOXILYZER SERIAL # 80-000312 LOCATION 1111 S LITCHFIELD RD

☒ 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- ☒ 1. Clock time check.
☒ 2. Date check.

OPERATIONAL TESTS

- ☒ 1. Alcohol-free subject test result 0. 000 AC.
☒ 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
☒ 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
☒ 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES ☒ NO ☐

COMMENTS

SIGNATURE [Signature]

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

01/29/2014

23:36:22

Standard Lot#: 13712100A2

Last Changed By: W. RYAN

#1100

QAS: M. GUILBAULT

#1144

GOODYEAR PD

Operator: M. GUILBAULT

#1144

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/1990

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:37:07
Diagnostic Test	Pass	23:37:42
Air Blank	0.000	23:38:10
0.100 Cal Check	0.101	23:38:30
Air Blank	0.000	23:39:01
Subject Test	0.000	23:39:14
Air Blank	RFI*	23:39:24
Air Blank	IPS**	23:39:31

*RFI Detect

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

01/29/2014

23:39:50

Standard Lot#: 13712100A2

Last Changed By: W. RYAN

#1100

QAS: M. GUILBAULT

#1144

GOODYEAR PD

Operator: M. GUILBAULT

#1144

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/1990

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:40:35
Diagnostic Test	Pass	23:41:10
Air Blank	0.000	23:41:37
0.100 Cal Check	0.100	23:41:58
Air Blank	0.000	23:42:28
Subject Test	XXX*	23:44:49
Air Blank	0.000	23:45:23

*Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

01/29/2014

23:51:03

Standard Lot#: 13712100A2

Last Changed By: W. RYAN

#1100

QAS: M. GUILBAULT

#1144

GOODYEAR PD

Operator: M. GUILBAULT

#1144

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/1990

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:51:59
Diagnostic Test	Pass	23:52:34
Air Blank	0.000	23:53:02
0.100 Cal Check	0.100	23:53:23
Air Blank	0.000	23:53:53
Subject Test	ABT*	23:54:02
Air Blank	0.000	23:54:30

*Sequence Aborted

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GURUBAUT AGENCY GOODYEAR PD
DATE 1/29/2014 TIME 2355
INTOXILYZER SERIAL # 80-000312 LOCATION 1111 S LOTCHHEAD Rd.

- ✓ () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 100 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1144

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-000312

Core Version: 8105.48

01/29/2014

23:55:57

Standard Lot#: 13712100A2

Last Changed By: W. RYAN #1100

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	23:56:27
0.100 Cal Check	0.100	23:56:47
Air Blank	0.000	23:57:18

Preventative Maintenance

Performed -- Timer Reset