

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. ROGERS AGENCY GOODYEAR P.D.
DATE 7-6-06 TIME 1032
INTOXILYZER SERIAL # 80-000312 LOCATION GOODYEAR P.D.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

LOT 432401I
EXP 11-23-06

SIGNATURE [Signature] #1123

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

07/06/2006

10:32:23

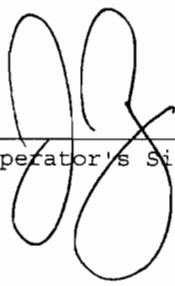
Standard Lot#: 432401I

Last Changed By: J. VALLANDINGHAM #981

QAS: J. ROGERS #1123

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	10:32:51
Cal Check	0.099	10:33:09
Air Blank	0.000	10:33:39

 #1123

Operator's Signature

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. ROGERS AGENCY GOODYEAR P.D.
DATE 7-6-06 TIME 1057
INTOXILYZER SERIAL # 80-000312 LOCATION GOODYEAR P.D.

(✓) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
(✓) 2. Date check.

LOT - 432401I
EXP - 11-23-06

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
(✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(✓) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES ☒ NO ☐

COMMENTS

WHEN ATTEMPTING THE DEFICIENT SAMPLE THE
RANGE EXCEEDED DUE TO TOO MUCH MOUTH
ALCOHOL. → MY MISTAKE NOT INSTRUMENT.

SIGNATURE [Signature] #1123

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

07/06/2006

10:35:58

Standard Lot#: 432401I

Last Changed By: J. VALLANDINGHAM #981

QAS: J. ROGERS

#1123

GOODYEAR PD

Operator: J. ROGERS

#1123

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:37:11
Diagnostic Test	Pass	10:37:38
Air Blank	0.000	10:38:05
0.100 Cal Check	0.098	10:38:24
Air Blank	0.000	10:38:54
Subject Test	0.000	10:39:12
Air Blank	ABT*	10:39:24
Air Blank	0.000	10:39:52

*Sequence Aborted

 #1123

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

07/06/2006

10:40:00

Standard Lot#: 432401I

Last Changed By: J. VALLANDINGHAM #981

QAS: J. ROGERS #1123
GOODYEAR PD

Operator: J. ROGERS #1123
GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	10:41:37
Air Blank	0.000	10:42:05

*Improper Sample

08 #1123

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

07/06/2006

10:42:53

Standard Lot#: 432401I

Last Changed By: J. VALLANDINGHAM #981

QAS: J. ROGERS #1123
GOODYEAR PD

Operator: J. ROGERS #1123
GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	10:43:34
Air Blank	0.000	10:44:03

*RFI Detect

 #1123

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

07/06/2006

10:44:42

Standard Lot#: 432401I

Last Changed By: J. VALLANDINGHAM #981

QAS: J. ROGERS #1123
GOODYEAR PD

Operator: J. ROGERS #1123
GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:45:33
Diagnostic Test	Pass	10:46:01
Air Blank	0.000	10:46:28
0.100 Cal Check	0.099	10:46:47
Air Blank	0.000	10:47:17
Subject Test	RGE*	10:47:35
Air Blank	0.000	10:48:14

*Range Exceeded

*08 #1123
TOO MUCH
MOUTH ALCOHOL.*

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

07/06/2006

10:49:03

Standard Lot#: 432401I

Last Changed By: J. VALLANDINGHAM #981

QAS: J. ROGERS #1123
GOODYEAR PD

Operator: J. ROGERS #1123
GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 100

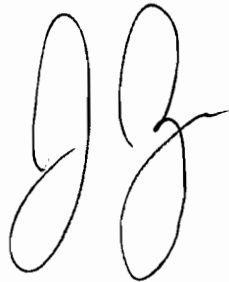
15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:49:52
Diagnostic Test	Pass	10:50:20
Air Blank	0.000	10:50:47
0.100 Cal Check	0.098	10:51:06
Air Blank	0.000	10:51:36
Subject Test	0.034*	10:54:44
Air Blank	0.000	10:55:15
Air Blank	0.000	10:55:43
Subject Test	XXX**	10:56:15
Air Blank	0.000	10:56:47

*Deficient Sample

**Invalid Sample

Begin new deprivation period

 #1123

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. ROGERS AGENCY GOODYEAR P.D.
DATE 7-6-06 TIME 1057
INTOXILYZER SERIAL # 80-000312 LOCATION GOODYEAR P.D.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0. 098 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove printed record. Attach the record to the completed checklist.

LOT 432401I
EXP 11-23-06

SIGNATURE [Signature] #1123

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

07/06/2006

10:57:50

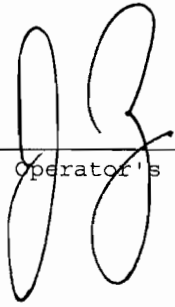
Standard Lot#: 432401I

Last Changed By: J. VALLANDINGHAM #981

QAS: J. ROGERS #1123

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	10:58:17
Cal Check	0.098	10:58:36
Air Blank	0.000	10:59:06

 #1123

Operator's Signature