EXHIBIT G-2 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPI	EC	CIALIST AG	ENCY GOODY BAR PO)	
DATE		, /		
INTOX	IL.	LYZER SERIAL # 86-003/2 LO	CATION GPD	
W	1.	. Ensure that gas tank is attached to instrument and conta	ins a standard alcohol concentration solution ./ DO AC	
		Pour a standard alcohol concentration solution simulator. Ensure that a tight seal has been made. Turn 0.2°C	AC, into a clean dry simulator and assemble the n on the simulator and allow temperature to reach $34^{\circ}C \pm$	
(Ur	2.	. Intoxilyzer 8000 display reads "PUSH BUTTON TO S	TART"	
	3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.			
(V	4.	A in blank completed	<i>a</i>	
ivi	5.	. Calibration check completed. Test results 009	7 _{AC.}	
المريخ	6.	. Air blank completed.		
		. Remove printed record. Attach the record to the compl	eted checklist.	
SIGNA		1		

DPS Form Exh G-2 (Rev 05-01)

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

07/11/2007

11:47:07

Standard Lot#: 618601I

Last Changed By: J. TRAINING #99999

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	11:47:34
Cal Check	0.099	11:47:53
Air Blank	0.000	11:48:22

Operator's Signature

EXHIBIT G-2 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SP	EC	IALIST W(U AGENCY GP)		
DATE		7/142007 TIME 1151		
INTOX	(IL	YZER SERIAL # 80 -000312 LOCATION GRO		
(W		Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution/_OAC.		
		Pour a standard alcohol concentration solutionAC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}C \pm 0.2^{\circ}C$		
(U)	2.	Intoxilyzer 8000 display reads "PUSH BUTTON TO START"		
(V)		Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.		
(V)	4.	Air blank completed.		
(\mathcal{V})	5.	Calibration check completed. Test results 0AC.		
(\mathcal{V})	6.	Air blank completed.		
(M)	7.	Remove printed record. Attach the record to the completed checklist.		
SIGNA	TU	JRE		
DPS F	orm	Exh G-2 (Rev 05-01)		

CHANGED GAS BOTTLE
706502 I
EX 3/9/2009

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

07/11/2007 11:51:46

Standard Lot#: 706502I


Last Changed By: W. WILL #974


QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	11:52:13
Cal Check	0.100	11:52:32
Air Blank	0.000	11:53:01

Operator's Signature

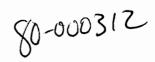

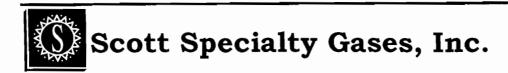

CERTIFICATE OF ANALYSIS EBSTM - Ethanol Breath Standard

Customer Name

CMI

316 East Ninth Street Owensboro, KY 42303 Assay Laboratory

ALM049986

Scott Specialty Gases

6141 Easton Road, Bldg. 1

Plumsteadville, Pa. 18949

Purchase Order: 370933 Scott Project # 0199359

Lot No.: 706502I Expiration Date: March 9, 2009

EXPIRATION

CYLINDER#

CONCENTRATION

DATE

05/18/2007

PPM 262.0

CONTAINER TYPE:105

REFERENCE STANDARD

N.I.S.T. Traceable Standards*

LOT CERTIFIED VALUE **CONCENTRATION**

COMPONENTS Ethanol Nitrogen

ppm (BrAC) 260.5 (0.100) Balance

ACCURACY

+/- 2%

*Certification: Traceable to N.I.S.T. RGM Ethanol Standards

Analyst:

Date: 03/14/2007

706502I