

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN 1100 AGENCY GOODYEAR POLICE DEPT

DATE 12-1-18 TIME 1551

INTOXILYZER SERIAL # 90-001858 LOCATION 11 N 145TH AVE, GOODYEAR AZ

(S) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 100 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan 1100

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

12/01/2018

15:51:48

Standard Lot#: 10818100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	15:52:44
0.100 Cal Check	0.100	15:53:04
Air Blank	0.000	15:53:35

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST W. T. Ryan 1100 AGENCY GOODYEAR POLICE DEPT
DATE 12-1-18 TIME 1611 HRS
INTOXILYZER SERIAL # 80-00185B LOCATION 11 N 145TH AVE, GOODYEAR AZ

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE W. T. Ryan 1100

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

12/01/2018

16:11:29

Standard Lot#: 10818100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 12/01/1959

Sex: M

Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	16:12:37
Diagnostic Test	Pass	16:13:12
Air Blank	0.000	16:13:39
0.100 Cal Check	0.100	16:14:00
Air Blank	0.000	16:14:30
Subject Test	0.000	16:14:49
Air Blank	0.000	16:15:17
Wait	ABT*	
Air Blank	0.000	16:15:55

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

12/01/2018

16:17:07

Standard Lot#: 10818100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 12/01/1959

Sex: M

Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	16:17:53
Air Blank	0.000	16:18:21

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

12/01/2018

16:21:02

Standard Lot#: 10818100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 12/01/1959

Sex: M Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	16:21:40
Air Blank	0.000	16:22:08

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

12/01/2018

16:23:32

Standard Lot#: 10818100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Subject: TEST,TEST

DOB: 12/01/1959

Sex: M

Weight: 185

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	16:24:26
Diagnostic Test	Pass	16:25:01
Air Blank	0.000	16:25:28
0.100 Cal Check	0.100	16:25:49
Air Blank	0.000	16:26:19
Subject Test	0.094*	16:29:27
Air Blank	0.000	16:30:08
Air Blank	0.000	16:30:36
Subject Test	XXX**	16:30:59
Air Blank	0.000	16:31:33

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

12/01/2018

16:32:47

Standard Lot#: 10818100A1

Last Changed By: W. RYAN #1100

QAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	16:33:16
0.100 Cal Check	0.099	16:33:37
Air Blank	0.000	16:34:07

Preventative Maintenance
Performed -- Timer Reset