

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST WLU AGENCY Goodyear PD
DATE 6/26/2009 TIME 10:52
INTOXILYZER SERIAL # 80-0312 LOCATION Goodyear PD

- W 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- W 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- W 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- W 4. Air blank completed.
- W 5. Calibration check completed. Test results 0. 101 AC.
- W 6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____

INTOXILYZER 8000

Location: GOODYEAR PD

Serial Number: 80-000312

06/26/2009

10:52:50

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT #1144

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time

Air Blank	0.000	10:53:17
Cal Check	0.101	10:53:36
Air Blank	0.000	10:54:06

Operator's Signature

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST WML AGENCY GOODYEAR PD
DATE 06/26/2009 TIME 10:57
INTOXILYZER SERIAL # 80-0312 LOCATION GOODYEAR PD

☒ 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- ☒ 1. Clock time check.
☒ 2. Date check.

OPERATIONAL TESTS

- ☒ 1. Alcohol-free subject test result 0. 000 AC.
☒ 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
☒ 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
☒ 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES ☒ NO ☐

COMMENTS

SIGNATURE Jh

INTOXILYZER 8000

Location: GOODYEAR PD

Serial Number : 80-000312

06/26/2009

10:57:29

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT #1144

QAS: W. WILL #974

GOODYEAR PD

Operator: W. WILL #974

GOODYEAR PD

Subject: TEST,TEST,T

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:58:33
Diagnostic Test	Pass	10:59:01
Air Blank	0.000	10:59:28
0.100 Cal Check	0.103	10:59:47
Air Blank	RFI*	10:59:58
Air Blank	0.000	11:00:26

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR PD

Serial Number : 80-000312

06/26/2009

11:01:00

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT #1144

QAS: W. WILL #974

GOODYEAR PD

Operator: W. WILL #974

GOODYEAR PD

Subject: TEST,TEST,A

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:02:06
Diagnostic Test	Pass	11:02:34
Air Blank	0.000	11:03:01
0.100 Cal Check	0.101	11:03:20
Air Blank	0.000	11:03:49
Subject Test	0.000	11:04:40
Air Blank	0.000	11:05:08
Five Minute Wait	ABT*	
Air Blank	0.000	11:05:47

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR PD

Serial Number : 80-000312

06/26/2009

11:06:15

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT #1144

QAS: W. WILL #974

GOODYEAR PD

Operator: W. WILL #974

GOODYEAR PD

Subject: TEST,TEST,T

DOB: 11/11/1111

Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	11:06:52
Air Blank	0.000	11:07:21

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR PD

Serial Number : 80-000312

06/26/2009

11:07:27

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT #1144

QAS: W. WILL #974
GOODYEAR PD

Operator: W. WILL #974
GOODYEAR PD

Subject: TEST,TEST,T

DOB: 11/11/1111

Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	11:08:28
Air Blank	0.000	11:08:56

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR PD

Serial Number : 80-000312

06/26/2009

11:09:02

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT #1144

QAS: W. WILL #974

GOODYEAR PD

Operator: W. WILL #974

GOODYEAR PD

Subject: TEST,TEST,T

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:10:01
Diagnostic Test	Pass	11:10:28
Air Blank	0.000	11:10:56
0.100 Cal Check	0.101	11:11:14
Air Blank	0.000	11:11:44
Subject Test	0.047*	11:14:52
Air Blank	0.000	11:15:25
Air Blank	0.000	11:15:52
Subject Test	XXX**	11:16:12
Air Blank	0.000	11:16:49

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR PD

Serial Number: 80-000312

06/26/2009

11:17:51

Standard Lot#: 808401G

Last Changed By: M. GUILBAULT #1144

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	11:18:18
Cal Check	0.100	11:18:37
Air Blank	0.000	11:19:06

Operator's Signature