

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR PD
DATE 3-19-2005 TIME 0306
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD BOOKING ROOM
QA SPECIALIST J. VALLANDINGHAM #981
(print name)

1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC.

OR

Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

4. Air blank completed.

5. Calibration check completed. Test results 0. 098 AC.

6. Air blank completed.

7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: _____

J. Vallandingham #981

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

03/19/2005

03:06:09


Standard Lot#: 422601G

Last Changed By: W. WILL #974

QAS: J. VALLANDINGHAM #981

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	03:06:37
Cal Check	0.098	03:06:56
Air Blank	0.000	03:07:25

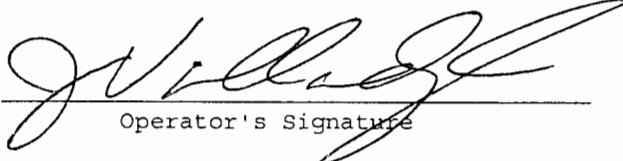

Operator's Signature