

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. ROGERS AGENCY GOODYEAR P.D.
DATE 3-22-07 TIME 1426
INTOXILYZER SERIAL # 80-000312 LOCATION GOODYEAR P.D.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

LOT 618601I
EXP 7/7/08

SIGNATURE [Signature] #1123

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

03/22/2007

14:26:26


Standard Lot#: 618601I

Last Changed By: J. ROGERS #1123

QAS: J. ROGERS #1123

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	14:26:53
Cal Check	0.099	14:27:12
Air Blank	0.000	14:27:41

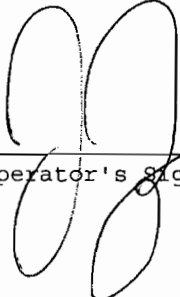

Operator's Signature #1123

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. ROGERS AGENCY GOODYEAR P.D.
DATE 3-22-07 TIME 1429
INTOXILYZER SERIAL # 80-000312 LOCATION GOODYEAR P.D.

(☒) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (☒) 1. Clock time check.
(☒) 2. Date check.

OPERATIONAL TESTS

- (☒) 1. Alcohol-free subject test result 0. 00 AC.
(☒) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(☒) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(☒) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES ☒ NO ☐

COMMENTS

SIGNATURE [Signature] #1123

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

03/22/2007

14:29:48

Standard Lot#: 618601I

Last Changed By: J. ROGERS

#1123

QAS: J. ROGERS

#1123

GOODYEAR PD

Operator: J. ROGERS

#1123

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:30:39
Diagnostic Test	Pass	14:31:07
Air Blank	0.000	14:31:34
0.100 Cal Check	0.100	14:31:53
Air Blank	0.000	14:32:22
Subject Test	0.000	14:32:39
Air Blank	0.000	14:33:06
Five Minute Wait	ABT*	
Air Blank	0.000	14:34:03

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

03/22/2007

14:36:31

Standard Lot#: 618601I

Last Changed By: J. ROGERS

#1123

QAS: J. ROGERS

#1123

GOODYEAR PD

Operator: J. ROGERS

#1123

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	14:37:08
Air Blank	0.000	14:37:37

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

03/22/2007

14:34:47

Standard Lot#: 618601I

Last Changed By: J. ROGERS

#1123

QAS: J. ROGERS

#1123

GOODYEAR PD

Operator: J. ROGERS

#1123

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	14:35:22
Air Blank	0.000	14:35:50

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

03/22/2007

14:38:16

Standard Lot#: 618601I

Last Changed By: J. ROGERS

#1123

QAS: J. ROGERS

#1123

GOODYEAR PD

Operator: J. ROGERS

#1123

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:39:07
Diagnostic Test	Pass	14:39:34
Air Blank	0.000	14:40:01
0.100 Cal Check	0.099	14:40:20
Air Blank	0.000	14:40:49
Subject Test	RGE*	14:41:14
Air Blank	0.000	14:41:51

*Range Exceeded

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

03/22/2007

14:42:37

Standard Lot#: 618601I

Last Changed By: J. ROGERS

#1123

QAS: J. ROGERS

#1123

GOODYEAR PD

Operator: J. ROGERS

#1123

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/0101

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:43:26
Diagnostic Test	Pass	14:43:53
Air Blank	0.000	14:44:20
0.100 Cal Check	0.100	14:44:39
Air Blank	0.000	14:45:09
Subject Test	0.031*	14:48:17
Air Blank	0.000	14:48:48
Air Blank	0.000	14:49:16
Subject Test	XXX**	14:49:45
Air Blank	0.000	14:50:18

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Rogers AGENCY GOODYEAR P.D.
DATE 3-22-07 TIME 1451
INTOXILYZER SERIAL # 80-000312 LOCATION GOODYEAR P.D.

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 098 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

LOT 618601E
EXP 7/7/08

SIGNATURE [Signature] #1123

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

03/22/2007

14:51:25


Standard Lot#: 618601I

Last Changed By: J. ROGERS #1123

QAS: J. ROGERS #1123

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	14:51:52
Cal Check	0.098	14:52:11
Air Blank	0.000	14:52:40

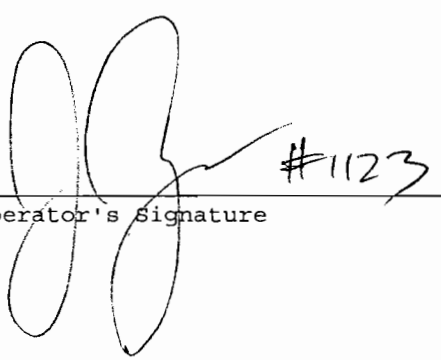

Operator's Signature