

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR PD
DATE 5-15-2005 TIME 1712
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD BOOKING STATION
QA SPECIALIST J. VALLANDINGHAM #981
(print name)

(✓) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC.

OR

Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

(✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

(✓) 4. Air blank completed.

(✓) 5. Calibration check completed. Test results 0. 098 AC.

(✓) 6. Air blank completed.

(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: _____



INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

05/15/2005

17:12:23

Standard Lot#: 422601G

Last Changed By: W. WILL #974

QAS: J. VALLANDINGHAM #981

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	17:12:50
Cal Check	0.098	17:13:09
Air Blank	0.000	17:13:38

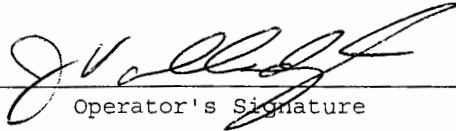

Operator's Signature

EXHIBIT 3

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

AGENCY GOODYEAR PD
DATE 5-15-2005 TIME 1723
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD BOOKING ROOM
QA SPECIALIST J. VALLANDINGHAM #981
(Print Name)

(☒) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

(☒) 1. Clock time check.

(☒) 2. Date check. _____

OPERATIONAL TESTS

(☒) 1. Alcohol-free subject test result 0. 000 AC.

(☒) 2. Error recognition logic system functioning.

Not a Successfully Completed Test Sequence printed

(☒) 3. Proper sample recognition system.

Not a Successfully Completed Test Sequence printed

Deficient sample printed.

(☒) 4. Calibration standard 0. 100 AC. Result 0. 096 AC.

Instrument is operating properly and accurately. YES ☒ NO _____

COMMENTS: _____

SIGNATURE J. Vallandingham

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

05/15/2005

17:23:55

Standard Lot#: 422601G

Last Changed By: W. WILL

#974

QAS: J. VALLANDINGHAM

#981

GOODYEAR PD

Operator: J. VALLANDINGHAM

#981

GOODYEAR PD

Subject: SQAP,TEST

DOB: 11/11/1111

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:24:53
Diagnostic Test	Pass	17:25:21
Air Blank	0.000	17:25:48
0.100 Cal Check	0.099	17:26:07
Air Blank	0.000	17:26:36
Subject Test	0.000	17:26:57
Air Blank	0.000	17:27:24
Five Minute Wait	ABT*	
Air Blank	0.000	17:28:01

*Sequence Aborted



Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

05/15/2005

17:28:23

Standard Lot#: 422601G

Last Changed By: W. WILL

#974

QAS: J. VALLANDINGHAM

#981

GOODYEAR PD

Operator: J. VALLANDINGHAM

#981

GOODYEAR PD

Subject: SQAP, TEST

DOB: 11/11/1111

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	17:29:06
Air Blank	0.000	17:29:35

*Improper Sample



Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

05/15/2005

17:29:58

Standard Lot#: 422601G

Last Changed By: W. WILL

#974

QAS: J. VALLANDINGHAM

#981

GOODYEAR PD

Operator: J. VALLANDINGHAM

#981

GOODYEAR PD

Subject: SQAP, TEST

DOB: 11/11/1111

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	17:30:47
Air Blank	0.000	17:31:16

*RFI Detect



Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

05/15/2005

17:31:39

Standard Lot#: 422601G

Last Changed By: W. WILL

#974

QAS: J. VALLANDINGHAM

#981

GOODYEAR PD

Operator: J. VALLANDINGHAM

#981

GOODYEAR PD

Subject: SQAP, TEST

DOB: 11/11/1111

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:32:45
Diagnostic Test	Pass	17:33:12
Air Blank	0.000	17:33:40
0.100 Cal Check	0.099	17:33:59
Air Blank	0.000	17:34:28
Subject Test	0.067*	17:37:36
Air Blank	0.000	17:38:09
Air Blank	0.000	17:38:36
Subject Test	XXX**	17:38:59
Air Blank	0.000	17:39:34

*Deficient Sample

**Invalid Sample

Begin new deprivation period



Not a Successfully Completed Test Sequence

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES


STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR PD
DATE 5-15-2005 TIME 1740
INTOXILYZER SERIAL NO 80-000312 LOCATION GOODYEAR PD BOOKING ROOM
QA SPECIALIST J. VALLANDINGHAM #981
(print name)

- (☒) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC.
- OR
- (☒) 2. Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (☒) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (☒) 4. Air blank completed.
- (☒) 5. Calibration check completed. Test results 0. 096 AC.
- (☒) 6. Air blank completed.
- (☒) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: _____

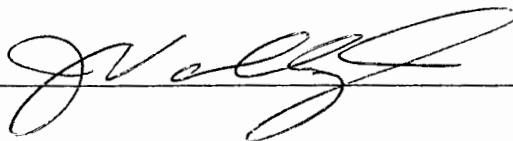

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

05/15/2005

17:40:39


Standard Lot#: 422601G

Last Changed By: W. WILL #974

QAS: J. VALLANDINGHAM #981

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	17:41:06
Cal Check	0.096	17:41:25
Air Blank	0.000	17:41:54

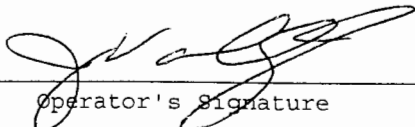

Operator's Signature