

EXHIBIT 2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR POLICE DEPARTMENT
DATE 5/11/2006 TIME 17:27
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD
QA SPECIALIST Will
(print name)

1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard 100 AC.
- OR
- Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 10/ AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: Sh

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

05/11/2006

17:27:52

Standard Lot#: 432401I

Last Changed By: J. VALLANDINGHAM #981

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	17:28:19
Cal Check	0.101	17:28:38
Air Blank	0.000	17:29:07

Operator's Signature