



**ARIZONA DEPARTMENT OF PUBLIC SAFETY**  
**INTOXILYZER SERVICE REQUEST**  
**SEND ALL COPIES TO DPS CRIME LABORATORY**

**QUALITY ASSURANCE SPECIALIST**

|  |  |                     |
|--|--|---------------------|
| DATE<br><b>5/16/07</b>                             | SERIAL NUMBER (List simulator S / N also)<br>INTOX: <b>80-00312</b> SIMULATOR: | FIELD LOCATION      |
| DPS EQUIPMENT NUMBER                               | SUBMITTING AGENCY'S FULL NAME (NO INITIALS)<br><b>GOOD/EAR PD</b>              |                     |
| QUALITY ASSURANCE SPECIALIST<br><b>WILL ROGERS</b> | PHONE NUMBER<br><b>(623) 882 7693</b>  |                     |
| QUALITY ASSURANCE SPECIALIST MAILING ADDRESS       |  | CITY STATE ZIP CODE |

Was a S.Q.A.P. completed prior to removal from service?

No, Explain: **NEEDS CAL ✓**

Yes, Date: \_\_\_\_\_ Date out of service: \_\_\_\_\_

No  Yes Do records need to be downloaded (ACTIVE C.O.B.R.A.. ONLY)

**EQUIPMENT SENT:**

- INTOXILYZER     POWER CORD     BREATH HOSE     EXTERNAL PRINTER     SIMULATOR     KEYBOARD     MODEM  
 HEATED SIMULATOR TUBE     BRACKET     OTHER **GAS TUBE CYLINDER**

PROBLEM DESCRIPTION: (What is the instrument doing or not doing? Include observations such as processor errors or other displayed messages.)

**UNIT WILL NOT POWERUP**

**LAB USE ONLY**

CORRECTIVE ACTION

**Replace power supply  
 verify CALIBRATION & FUNCTION CHECK**

COMMENTS

TECHNICIAN'S NAME

CALLED FOR PICKUP

BADGE / ID NO. TITLE

CONTACT PERSON

DATE

**[Signature]**      **6101 Elec tech**      **5/18/07**

INTOXILYZER 8000  
Location: GOODYEAR POLICE DEPT  
Serial Number: 80-000312  
05/18/2007 09:58:22

-----  
Standard Lot#: 6186011  
Last Changed By: J. TRAINING #99999

-----  
OAS: J. ROGERS #1123  
GOODYEAR PD

-----  
Operator: J. TRAINING #99999  
AZ DPS

-----  
Subject: TWENTY FOUR, SAMPLE, T  
DOB: 03/01/1965  
Sex: M Weight: 165

-----  
15 Minute Deprivation Period? Yes

-----  
Test g/210L Time  
-----  
Air Blank 0.000 09:59:04  
Diagnostic Test Pass 09:59:33  
Air Blank 0.000 10:00:00  
0.100 Cal Check 0.100 10:00:19  
Air Blank 0.000 10:00:49  
Subject Test 0.198 10:01:45  
Air Blank 0.000 10:02:18  
Five Minute Wait  
Air Blank 0.000 10:07:13  
Subject Test 0.192 10:07:46  
Air Blank 0.000 10:08:18  
0.100 Cal Check 0.098 10:08:37  
Air Blank 0.000 10:09:06  
Diagnostic Test Pass 10:09:35

Successfully  
Completed  
Test Sequence

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. VALLANDINGHAM #981 AGENCY GOODYEAR PD  
DATE 5-18-2007 TIME 13:31  
INTOXILYZER SERIAL # 80-00312 LOCATION GOODYEAR PD - MAIN STATION

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 099 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE \_\_\_\_\_

DPS Form Exh G-2 (Rev 05-01)

INTOXILYZER 8000  
Location: GOODYEAR POLICE DEPT  
Serial Number: 80-00312  
05/18/2007 13:31:07

Standard Lot#: 6186011  
Last Changed By: J. TRAINING #99999

QAS: J. VALLANDINGHAM #981  
GOODYEAR PD

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | 0.000  | 13:31:34 |
| Cal Check | 0.099  | 13:31:53 |
| Air Blank | 0.000  | 13:32:22 |

J. Vallandingham #981  
Operator's Signature