

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GERSHAWT 1144 AGENCY GOODYEAR PD  
DATE 5/10/2011 TIME 1331  
INTOXILYZER SERIAL # 80-000312 LOCATION 10701 S ESTRELLA PKWY

P. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 102 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1144

INTOXILYZER 8000  
Location: 10701 S ESTRELLA  
Serial Number: 80-000312  
Core Version: 8105.48  
05/10/2011

13:31:50

-----  
Standard Lot#: 767153  
Last Changed By: M. GUILBAULT #1144  
-----

QAS: M. GUILBAULT #1144  
GOODYEAR PD  
-----

Operator: M. GUILBAULT #1144  
GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	13:32:58
0.100 Cal Check	0.102	13:33:18
Air Blank	0.000	13:33:48

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GUAYAN 1144 AGENCY GOODYEAR PA  
DATE 5/10/201 TIME 1337  
INTOXILYZER SERIAL 0-000312 LOCATION 10701 S ESTRELLA PKWY

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 101 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1144

INTOXILYZER 8000  
Location: 10701 S ESTRELLA  
Serial Number: 80-000312  
Core Version: 8105.48  
05/10/2011

13:37:02

*[Handwritten signature]*  
1144

-----  
Standard Lot#: 01011100A1  
Last Changed By: M. GUILBAULT #1144  
-----

QAS: M. GUILBAULT #1144  
GOODYEAR PD  
-----

Operator: M. GUILBAULT #1144  
GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	13:37:32
0.100 Cal Check	0.101	13:37:53
Air Blank	0.000	13:38:23

Preventative Maintenance  
Performed -- Timer Reset



7 Eastgate Dr. • P.O. Box 790 • Jacksonville, IL 62651-0790  
217-245-2183 • Fax: 217-243-7634 • www.ilmoproducts.com

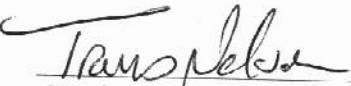
## Certificate of Analysis

**Certificate ID:** 1184  
**Part #:** BAC105L100T  
**Cylinder Size:** 105L  
**Lot Number:** 01011100A1  
**Expiration:** 2/1/2013

### 0.100 BAC (For use with breath alcohol testing instruments)

**Contents:** 105 Liters @ 1025 psig 70°F (21°C)

Component:	Concentration:	Accuracy:	Method:
Ethanol	260.5 ppm	+/- 0.002 or 2%	NDIR
Nitrogen	Balance	BAC whichever is greater	

  
Specialty Gas Lab Manager

1-17-2011  
Date

Distributed by: CMI Inc.  
316 East Ninth Street  
Owensboro, KY 42303  
Phone 866-835-0690  
[www.alcoholtest.com](http://www.alcoholtest.com)



ISO/IEC 17025:2005 Accredited Laboratory



7 Eastgate Dr. • P.O. Box 790 • Jacksonville, IL 62650  
 217-245-2183 • 888-243-9353 • www.ilmospecgas.com

## MATERIAL SAFETY DATA SHEET

This Material Safety Data Sheet (MSDS) complies with the requirements of OSHA's Hazard Communication Standard.

### NON-FLAMMABLE GAS MIXTURE

#### SECTION 1 - PRODUCT IDENTIFICATION

<b>Product Name/ Class</b>	Non-Flammable Gas mixture
<b>Product Number</b>	BAC DSP 0001
<b>Manufacturer</b>	ILMO Products, 7 Eastgate Drive, Jacksonville, IL 62651

#### SECTION 2 - HAZARDOUS INGREDIENTS

Ingredients	CAS Number	Percent	Exposure Limits	
			TLV	PEL
Ethanol	64-17-5	<14.8 (1-500 ppm)	1000, A4 (Not Classable as a human carcinogen)	TWA = 500
Nitrogen	7727-37-9	Balance	Simple Asphyxiant	

NE= Not Established

#### SECTION 3 - PHYSICAL CHARACTERISTICS

Boiling Point -320.4° F; -195.8° C	Specific Gravity (air =1): 0.967	Solubility in Water: 1.49% (v/v)
Vapor Pressure (psia): N/A	Melting/Freezing Point -210° C; -345.8° F	% Volatile: N/A
Vapor Density: 1.145 kg/m³	Evaporation Rate (Butyl Acetate =1) N/A	Appearance and Odor: Colorless gas with an alcohol odor.

#### SECTION 4 - FIRE and EXPLOSION HAZARD DATA

Flash Point (Method Used): N/A	Flammable Limits:	LEL: N/A UEL: N/A
-----------------------------------	-------------------	----------------------

**Extinguishing Media:** Non-flammable. Use extinguishing media appropriate for surrounding fire.

**Special Fire Fighting Procedures:** Structural fire-fighters must wear Self Contained Breathing Apparatus and full protective equipment.

**Unusual Fire and Explosion Hazards:** This product does not burn; however, containers, when involved in fire, may rupture or burst in the heat of the fire.

**Explosion Sensitivity to Mechanical Impact:** Not Sensitive.

**Explosion Sensitivity to Static Discharge:** Not Sensitive.

#### SECTION 5 - REACTIVITY DATA

Stability	Unstable <input type="checkbox"/>	Stable <input checked="" type="checkbox"/>	Conditions to Avoid: Contact with incompatible materials: Cylinders exposed to high temperatures of direct flame can rupture or bust.
<b>Incompatibility (Materials to Avoid):</b> Nitrogen, the main component is not compatible with Titanium.			
<b>Hazardous Decomposition or Byproducts:</b> None.			
Hazardous Polymerization	May Occur <input type="checkbox"/>	Will Not Occur <input checked="" type="checkbox"/> *	*Hazardous polymerization will not occur under normal circumstances.

#### SECTION 6 - HEALTH HAZARD DATA

Routes of Entry:	<input checked="" type="checkbox"/> Inhalation	<input type="checkbox"/> Skin	<input type="checkbox"/> Ingestion
------------------	------------------------------------------------	-------------------------------	------------------------------------

**Health Hazards (Acute and Chronic):** ACUTE: The most significant hazard associated with this gas is inhalation of Oxygen-deficient atmospheres. Symptoms of Oxygen deficiency includes respiratory difficulty, ringing in the ears, headaches, shortness of breath, wheezing, headache, dizziness, indigestion, nausea, and, at high concentrations, unconsciousness or death may occur. The skin of the victim of over-exposure may have a blue color. Contact with rapidly expanding gases (which released under high pressure) may cause frost bite. Symptoms of frostbite include change in skin color to white or grayish-yellow. The pain after contact with liquid can quickly subside.

**Carcinogenicity:** NTP: N/A IARC: N/A OSHA Regulated: N/A

**Signs and Symptoms of Exposure:** High concentrations of this gas can cause an Oxygen-deficient environment. Individuals breathing such an atmosphere may experience symptoms which include headaches, ringing in ears, dizziness, drowsiness, unconsciousness, nausea, vomiting and depression of all the senses. Under some circumstances, death may occur

**Medical Conditions Generally Aggravated by Exposure:** Pre-existing respiratory conditions may be aggravated by the over-exposure of this product

**Emergency and First Aid Procedures:** RESCUERS SHOULD NOT ATTEMPT TO REVIVE VICTIMS OF EXPOSURE TO THIS PRODUCT WITHOUT ADEQUATE PERSONAL PROTECTIVE EQUIPMENT. At a minimum, Self-Contained Breathing Apparatus and Fire-Retardant Personal Protective equipment should be worn. Adequate fire protections must be provided during rescue situations. Remove victim(s) to fresh air as quickly as possible. Trained personnel should administer supplemental Oxygen and/or cardio-pulmonary resuscitation, if necessary. Only trained personnel should administer supplemental Oxygen. Victim(s) must be taken for medical attention. Rescuers should be taken for medical attention, if necessary. Take copy of label and MSDS to physician or other health professional with victim(s).