



Scott Specialty Gases, Inc.

CERTIFICATE OF ANALYSIS

EBS™ - Ethanol Breath Standard

Customer Name
CMI
316 East Ninth Street
Owensboro, KY 42303

Assay Laboratory
Scott Specialty Gases
6141 Easton Road, Bldg. 1
Plumsteadville, Pa.
18949-0310

Purchase Order :365722
Scott Project #: 0195092
Lot No.: 3218021
Expiration Date:
August 12, 2005

REFERENCE STANDARD

N.I.S.T. Traceable Standards*

CYLINDER

ALM-060154
ALM-060204
ALM-059962
ALM-037472

CONCENTRATION ppm

103.6
118.1
222.2
251.0

CONTAINER TYPE: 105

LOT CERTIFIED VALUE

CONCENTRATION

COMPONENTS

Ethanol
Nitrogen

ppm (BrAC)

260.5 (0.100)
Balance

ACCURACY

+/- 2%

*Certification: Traceable to N.I.S.T. RGM Ethanol Standards

Analyst:

Floyd Bards

Date: 08/12/2003

EXHIBIT 3
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

AGENCY GOODYEAR Police Dept
DATE 11/14/2003 TIME 1129
INTOXILYZER SERIAL NO. 80-00312 LOCATION GLENDALE PD
QA SPECIALIST Will
(Print Name)

2) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (X) 1. Clock time check.
(X) 2. Date check.

OPERATIONAL TESTS

- (X) 1. Alcohol-free subject test result 0. 000 AC.
(X) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(X) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(X) 4. Calibration standard 0. 0.100% AC. Result 0. .097% AC.

Instrument is operating properly and accurately. YES (X) NO _____

COMMENTS: _____

SIGNATURE Will

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

11/14/2003

11:06:11

Standard Lot#: 321802I

Last Changed By: T. ONLY

#00

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: SQAP,TEST

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:07:11
Diagnostic Test	Pass	11:07:37
Air Blank	0.000	11:08:04
0.100 Cal Check	0.097	11:08:23
Air Blank	0.000	11:08:52
Subject Test	0.000	11:09:09
Air Blank	ABT*	11:09:17
Air Blank	RFI**	11:09:24

*Sequence Aborted

**RFI Detect

**RFI DETECTED IN ROOM
REDID TEST*

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

11/14/2003

11:11:48

Standard Lot#: 321802I

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: SQAP,TEST

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	ABT*	11:12:26
Air Blank	0.000	11:12:53

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

11/14/2003

11:13:15

Standard Lot#: 321802I

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: SQAP, TEST

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	11:13:58
Air Blank	0.000	11:14:26

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

11/14/2003

11:14:46

Standard Lot#: 321802I

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: SQAP,TEST

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:15:38
Diagnostic Test	Pass	11:16:05
Air Blank	0.000	11:16:32
0.100 Cal Check	0.097	11:16:50
Air Blank	0.000	11:17:19
Subject Test	0.035*	11:20:27
Air Blank	0.000	11:20:59
Air Blank	0.000	11:21:25
Subject Test	0.153	11:22:32
Air Blank	ABT**	11:22:42
Air Blank	0.000	11:23:10

*Deficient Sample

**Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

11/14/2003

11:23:17

Standard Lot#: 321802I

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: SQAP,TEST

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	11:23:53
Air Blank	0.000	11:24:20

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-000312

11/14/2003

11:24:32

Standard Lot#: 321802I

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: SQAP,TEST

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:25:26
Diagnostic Test	Pass	11:25:53
Air Blank	0.000	11:26:20
0.100 Cal Check	0.097	11:26:38
Air Blank	0.000	11:27:07
Subject Test	XXX*	11:27:28
Air Blank	0.000	11:28:02

*Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

11/14/2003

11:28:56

Standard Lot#: 321802I

Last Changed By: W. WILL #974

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
-----	-----	-----
Air Blank	0.000	11:29:22
Cal Check	0.097	11:29:41
Air Blank	0.000	11:30:10

Operator's Signature

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR POLICE DEPT
DATE 11/14/2003 TIME 1104
INTOXILYZER SERIAL NO. 80-00312 LOCATION GLendale PD
QA SPECIALIST Will
(print name)

- (1) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard 0.100 AC.
- OR
- Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (2) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (3) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (4) 4. Air blank completed.
- (5) 5. Calibration check completed. Test results 0. 097 AC.
- (6) 6. Air blank completed.
- (7) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: [Signature]

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

11/14/2003

11:04:07

Standard Lot#: 321802I

Last Changed By: T. ONLY #00

QAS: T. ONLY #00

TRAINING

Test g/210L Time

Air Blank 0.000 11:04:34
Cal Check 0.097 11:04:52
Air Blank 0.000 11:05:21

Operator's Signature

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)


ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY Goodyear PD
DATE 11/29/2003 TIME 1706
INTOXILYZER SERIAL NO. 80-000312 LOCATION 6PD
QA SPECIALIST Will
(print name)

- (u) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC.
- OR
- Pour a standard alcohol solution of known value, _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (u) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (u) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (u) 4. Air blank completed.
- (u) 5. Calibration check completed. Test results 0. 100 AC.
- (u) 6. Air blank completed.
- (u) 7. Remove printed record. Attach the record to the completed checklist.

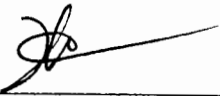SIGNATURE: 

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

11/29/2003

17:06:12


Standard Lot#: 321802I

Last Changed By: W. WILL #974

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	17:06:39
Cal Check	0.100	17:06:57
Air Blank	0.000	17:07:26

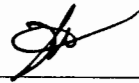

Operator's Signature