

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN 1100 AGENCY GOODYEAR POLICE DEPT  
DATE 5-1-2019 TIME 0002 HRS  
INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE GOODYEAR AZ

(S) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0-100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (S) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(S) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(S) 4. Air blank completed.  
(S) 5. Calibration check completed. Test results 0. 099 AC.  
(S) 6. Air blank completed.  
(S) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan #1100

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/01/2019

00:02:39

-----  
Standard Lot#: 10818100A1

Last Changed By: m. guilbault #1144  
-----

QAS: W. RYAN #1100

GOODYEAR PD  
-----

Operator: W. RYAN #1100

GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	00:03:42
0.100 Cal Check	0.099	00:04:03
Air Blank	0.000	00:04:33

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W.T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT

DATE 5-14-19 TIME 1915 HRS

INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE, GOODYEAR

AZ  
65338

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0.106 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W.T. Ryan #1100

DPS Form Exh G-2 (Rev 05-01)

TAKEN OUT OF SVE FOR DPS  
YEARLY MAINT.

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-001858  
Core Version: 8105.48

05/14/2019 19:15:29

-----  
Standard Lot#: 10818100A1  
Last Changed By: m. guilbault #1144

-----  
QAS: w. ryan #1100  
goodyear pd

-----  
Operator: w. ryan #1100  
goodyear pd

-----  
Test g/210L Time  
-----  
Air Blank 0.000 19:16:26  
0.100 Cal Check 0.100 19:16:46  
Air Blank 0.000 19:17:17

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST W.T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT  
DATE 5-14-19 TIME 1919 HRS

INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE, GOODYEAR AZ 85338

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.  
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.  
 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

TAKEN OUT OF SERVICE FOR DPS MAINT

SIGNATURE W.T. Ryan #1100

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/14/2019

19:19:27

---

Standard Lot#: 10818100A1

Last Changed By: m. guilbault #1144

---

QAS: w. ryan #1100  
goodyear pd

---

Operator: w. ryan #1100  
goodyear pd

---

Subject: TEST,TEST

DOB: 05/14/1959

Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	19:20:30
Diagnostic Test	Pass	19:21:05
Air Blank	0.000	19:21:32
0.100 Cal Check	0.100	19:21:53
Air Blank	0.000	19:22:23
Subject Test	0.000	19:22:54
Air Blank	0.000	19:23:21
Wait	ABT*	
Air Blank	0.000	19:24:01

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/14/2019

19:24:26

---

Standard Lot#: 10818100A1

Last Changed By: m. guilbault #1144

---

QAS: w. ryan #1100  
goodyear pd

---

Operator: w. ryan #1100  
goodyear pd

---

Subject: TEST,TEST

DOB: 05/14/1959

Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	19:25:09
Air Blank	0.000	19:25:38

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/14/2019

19:26:03

---

Standard Lot#: 10818100A1

Last Changed By: m. guilbault #1144

---

QAS: w. ryan #1100  
goodyear pd

---

Operator: w. ryan #1100  
goodyear pd

---

Subject: TEST,TEST

DOB: 05/14/1959

Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	19:26:48
Air Blank	0.000	19:27:16

\*RFI Detect

**Not a Successfully Completed Test Sequence**



INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/14/2019

19:27:34

---

Standard Lot#: 10818100A1

Last Changed By: m. guilbault #1144

---

QAS: w. ryan #1100  
goodyear pd

---

Operator: w. ryan #1100  
goodyear pd

---

Subject: TEST,TEST

DOB: 05/14/1959

Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	19:28:31
Diagnostic Test	Pass	19:29:06
Air Blank	0.000	19:29:33
0.100 Cal Check	0.100	19:29:54
Air Blank	0.000	19:30:24
Subject Test	0.126*	19:33:31
Air Blank	0.000	19:34:04
Air Blank	0.000	19:34:31
Subject Test	XXX**	19:34:58
Air Blank	0.000	19:35:33

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/14/2019

19:36:09

-----  
Standard Lot#: 10818100A1

Last Changed By: m. guilbault #1144  
-----

QAS: w. ryan #1100

goodyear pd  
-----

Operator: w. ryan #1100

goodyear pd  
-----

Test	g/210L	Time
Air Blank	0.000	19:36:41
0.100 Cal Check	0.099	19:37:01
Air Blank	0.000	19:37:32

Preventative Maintenance  
Performed -- Timer Reset



ARIZONA DEPARTMENT OF PUBLIC SAFETY

INTOXILYZER MAINTENANCE REPORT

CR NO. 1800018580008

Page 1 of 1

AGENCY Goodyear Police Department  
Goodyear, AZ 85338  
OFFICER RYAN, #1100  
DATE May 20, 2019



**INSTRUMENT**

80001858

**INSTRUMENT LOCATION**

Goodyear Police Department

**MAINTENANCE REQUESTED**

Repair and Preventative Maintenance

**MAINTENANCE PERFORMED**

***Parts Replaced:***

Breath Hose, End Block

***Preventative Maintenance Performed:***

Inspection, Calibration, Verification

*David K. Worthen*

---

DAVID K. WORTHEN, #4871, Forensic Scientist  
Central Regional Crime Laboratory  
2102 W. Encanto Blvd., Phoenix, AZ 85009  
602-223-2415

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W. T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT

DATE 5-21-19 TIME 1706 HRS

INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE, GOODYEAR AZ 85338

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.100 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W. T. Ryan #1100

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-001858  
Core Version: 8105.48  
05/21/2019 17:06:22

-----  
Standard Lot#: 10818100A1  
Last Changed By: m. guilbault #1144  
-----

DAS: W. RYAN #1100  
GOODYEAR PD  
-----

Operator: W. RYAN #1100  
GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	17:06:53
0.100 Cal Check	0.100	17:07:14
Air Blank	0.000	17:07:43

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST W.T. Ryan #1100 AGENCY Goodyear Police Dept

DATE 5-21-19 TIME 1711 HRS

INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE, GOODYEAR AZ

1. Display Reads "PUSH BUTTON TO START" GSB38

DIAGNOSTIC TESTS

- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC.
- 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

BROUGHT BACK INTO SERVICE AFTER  
DPS MAINT.

SIGNATURE W.T. Ryan #1100

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/21/2019

17:11:02

---

Standard Lot#: 10818100A1

Last Changed By: m. guilbault #1144

---

QAS: W. RYAN #1100

GOODYEAR PD

---

Operator: W. RYAN #1100

GOODYEAR PD

---

Subject: TEST,TEST

DOB: 05/21/1959

Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	17:11:58
Diagnostic Test	Pass	17:12:33
Air Blank	0.000	17:13:00
0.100 Cal Check	0.100	17:13:21
Air Blank	0.000	17:13:50
Subject Test	0.000	17:14:05
Air Blank	0.000	17:14:33
Wait	ABT*	
Air Blank	0.000	17:15:08

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/21/2019

17:16:06

---

Standard Lot#: 10818100A1

Last Changed By: m. guilbault #1144

---

QAS: W. RYAN #1100

GOODYEAR PD

---

Operator: W. RYAN #1100

GOODYEAR PD

---

Subject: TEST,TEST

DOB: 05/21/1959

Sex: M Weight: 185

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	17:16:44
Air Blank	0.000	17:17:12

\*Improper Sample

**Not a Successfully Completed Test Sequence**



INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/21/2019

17:17:35

---

Standard Lot#: 10818100A1

Last Changed By: m. guilbault #1144

---

QAS: W. RYAN #1100

GOODYEAR PD

---

Operator: W. RYAN #1100

GOODYEAR PD

---

Subject: TEST,TEST

DOB: 05/21/1959

Sex: M Weight: 185

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	17:18:11
Air Blank	0.000	17:18:40

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

05/21/2019

17:23:01

---

Standard Lot#: 10818100A1

Last Changed By: m. guilbault #1144

---

QAS: W. RYAN #1100

GOODYEAR PD

---

Operator: W. RYAN #1100

GOODYEAR PD

---

Subject: TEST,TEST

DOB: 05/21/1959

Sex: M Weight: 185

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	17:23:56
Diagnostic Test	Pass	17:24:31
Air Blank	0.000	17:24:58
0.100 Cal Check	0.099	17:25:19
Air Blank	0.000	17:25:49
Subject Test	0.040*	17:28:56
Air Blank	0.000	17:29:27
Air Blank	0.000	17:29:55
Subject Test	XXX**	17:31:51
Air Blank	0.000	17:32:26

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: 11 N 145TH AVE  
Serial Number: 80-001858  
Core Version: 8105.48  
05/21/2019 17:33:50

-----  
Standard Lot#: 10818100A1  
Last Changed By: m. guilbault #1144  
-----

QAS: W. RYAN #1100  
GOODYEAR PD  
-----

Operator: W. RYAN #1100  
GOODYEAR PD  
-----

Test	g/210L	Time
Air Blank	0.000	17:34:19
0.100 Cal Check	0.098	17:34:40
Air Blank	0.000	17:35:10

Preventative Maintenance  
Performed -- Timer Reset