

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR POLICE DEPARTMENT  
DATE 10/1/2004 TIME 16:05  
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD  
QA SPECIALIST Will  
(print name)

- (u) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC.

OR

Pour a standard alcohol solution of known value, \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (u) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

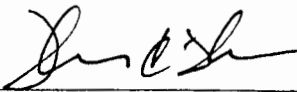
- (u) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

- (u) 4. Air blank completed.

- (u) 5. Calibration check completed. Test results 0. .098 AC.

- (u) 6. Air blank completed.

- (u) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: 

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

10/01/2004

16:05:16

-----  
Standard Lot#: 325402G

Last Changed By: W. WILL #974  
-----

QAS: W. WILL #974

GOODYEAR PD  
-----

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | 0.000  | 16:05:44 |
| Cal Check | 0.098  | 16:06:02 |
| Air Blank | 0.000  | 16:06:32 |

\_\_\_\_\_  
Operator's Signature

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

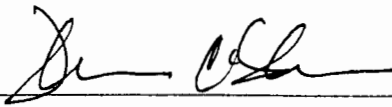
ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR POLICE DEPARTMENT  
DATE 10/22/2004 TIME 20:30  
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD  
QA SPECIALIST Will  
( print name)

- (u) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard - 100 AC.
- OR
- Pour a standard alcohol solution of known value , \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (u) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (u) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (u) 4. Air blank completed.
- (u) 5. Calibration check completed. Test results 0. 098 AC.
- (u) 6. Air blank completed.
- (u) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: 

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

10/22/2004

20:30:43

-----  
Standard Lot#: 325402G

Last Changed By: W. WILL #974  
-----

QAS: W. WILL #974

GOODYEAR PD  
-----

| Test      | g/210L | Time     |
|-----------|--------|----------|
| -----     |        |          |
| Air Blank | 0.000  | 20:31:11 |
| Cal Check | 0.098  | 20:31:29 |
| Air Blank | 0.000  | 20:31:59 |

\_\_\_\_\_  
Operator's Signature

EXHIBIT 2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

AGENCY GOODYEAR POLICE DEPARTMENT  
DATE 10/30/2004 TIME 16:41  
INTOXILYZER SERIAL NO. 80-000312 LOCATION GOODYEAR PD  
QA SPECIALIST WILL  
( print name)

- (u) 1. Ensure dry gas tank is attached to instrument and contains a known alcohol standard .100 AC.
- OR
- Pour a standard alcohol solution of known value, \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (u) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (u) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (u) 4. Air blank completed.
- (u) 5. Calibration check completed. Test results 0. .098 AC.
- (u) 6. Air blank completed.
- (u) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE: J. C. S.

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-000312

10/30/2004

16:41:24

-----  
Standard Lot#: 325402G

Last Changed By: W. WILL #974

-----  
QAS: W. WILL #974

GOODYEAR PD  
-----

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | 0.000  | 16:41:52 |
| Cal Check | 0.098  | 16:42:10 |
| Air Blank | 0.000  | 16:42:40 |

\_\_\_\_\_  
Operator's Signature