

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST W. T. RYAN #1100 AGENCY GOODYEAR POLICE DEPT

DATE 7-3-2019 TIME 2253 HRS

INTOXILYZER SERIAL # 80-000312 LOCATION 11 N 145TH AVE, GOODYEAR AZ 85338

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0-100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 098 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE W. T. Ryan #1100

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-000312

Core Version: 8105.48

07/03/2019

22:53:42

Standard Lot#: 10818100A1

Last Changed By: W. RYAN #1100

DAS: W. RYAN #1100

GOODYEAR PD

Operator: W. RYAN #1100

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	22:54:14
0.100 Cal Check	0.098	22:54:34
Air Blank	0.000	22:55:05

Preventative Maintenance
Performed -- Timer Reset