

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. ROGERS AGENCY GOODYEAR P.D.
DATE 8-14-07 TIME 1026
INTOXILYZER SERIAL # 80-001858 LOCATION GOODYEAR P.D.

☒ 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

☒ 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

☒ 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

☒ 4. Air blank completed.

☒ 5. Calibration check completed. Test results 0. 098 AC.

☒ 6. Air blank completed.

☒ 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____

DPS Form Exh G-2 (Rev 05-01)

BACK IN SERVICE
FROM OPS

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

08/14/2007

10:26:47


Standard Lot#: 536401I

Last Changed By: M. SLONEKER #6559

QAS: J. ROGERS #1123

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	10:27:14
Cal Check	0.098	10:27:33
Air Blank	0.000	10:28:02

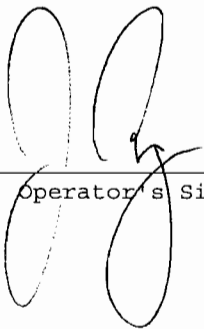

Operator's Signature

BACK IN SERVICE
FROM OPS.