

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GUELBAULT 1144 AGENCY GOODYEAR PD
DATE 8/21/2012 TIME 1741
INTOXILYZER SERIAL # 80-001858 LOCATION 10701 S ESTRELLA PKWY

- ☒ 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- ☒ 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ☒ 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ☒ 4. Air blank completed.
- ☒ 5. Calibration check completed. Test results 0. 100 AC.
- ☒ 6. Air blank completed.
- ☒ 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____

INTOXILYZER 8000
Location: 10701 S ESTRELLA PKY
Serial Number: 80-001858
Core Version: 8105.48
08/21/2012

17:41:02

Standard Lot#: 34011100A1
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	17:41:42
0.100 Cal Check	0.100	17:42:03
Air Blank	0.000	17:42:32

Preventative Maintenance
Performed -- Timer Reset