

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GURUBANT 1144 AGENCY GOODYEAR PD
DATE 8/8/13 TIME 2218
INTOXILYZER SERIAL # 80-001858 LOCATION 1111 S LECATSEED RD

(*) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (*) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(*) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(*) 4. Air blank completed.
(*) 5. Calibration check completed. Test results 0. 099 AC.
(*) 6. Air blank completed.
(*) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

08/08/2013

22:18:25

Standard Lot#: 34011100A1

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	22:18:54
0.100 Cal Check	0.099	22:19:15
Air Blank	0.000	22:19:45

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. G. B. M. S. T. AGENCY GOODYEAR PD
DATE 8/15/13 TIME 2235
INTOXILYZER SERIAL # 80-001858 LOCATION 1111 S VICTORIA RD

- (10) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (10) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (10) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (10) 4. Air blank completed.
- (10) 5. Calibration check completed. Test results 0. 099 AC.
- (10) 6. Air blank completed.
- (10) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

08/15/2013

22:35:09

Standard Lot#: 34011100A1

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	22:35:39
0.100 Cal Check	0.099	22:36:00
Air Blank	0.000	22:36:29

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. GURMANIT 1144 AGENCY GOOD YEAR PD
DATE 8/15/13 TIME 2230
INTOXILYZER SERIAL # 80-001858 LOCATION 111 S LITCHFIELD RD

(☒) Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

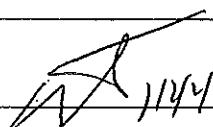
- (☒) 1. Clock time check.
(☒) 2. Date check.

OPERATIONAL TESTS

- (☒) 1. Alcohol-free subject test result 0. φ AC.
(☒) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(☒) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(☒) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES ☒ NO ☐

COMMENTS

SIGNATURE  1144

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

08/15/2013

22:36:37

Standard Lot#: 34011100A1

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Subject: TEST,TEST

DOB: 10/10/1990

Sex: M

Weight: 190

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	22:37:21
Diagnostic Test	Pass	22:37:56
Air Blank	0.000	22:38:23
0.100 Cal Check	0.099	22:38:44
Air Blank	0.000	22:39:13
Subject Test	0.000	22:39:30
Air Blank	IPS*	22:39:37
Air Blank	RFI**	22:39:44

*Improper Sample

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

08/15/2013

22:39:49

Standard Lot#: 34011100A1

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Subject: TEST,TEST

DOB: 10/10/1990

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	22:40:38
Diagnostic Test	Pass	22:41:13
Air Blank	0.000	22:41:40
0.100 Cal Check	0.099	22:42:01
Air Blank	0.000	22:42:31
Subject Test	0.048*	22:45:38
Air Blank	0.000	22:46:10
Air Blank	0.000	22:46:38
Subject Test	XXX**	22:47:01
Air Blank	0.000	22:47:34

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

08/15/2013

22:47:37

Standard Lot#: 34011100A1

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Subject: TEST,TEST

DOB: 10/10/1990

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	22:48:25
Diagnostic Test	Pass	22:49:00
Air Blank	0.000	22:49:28
Reference	ABT*	22:49:33
Air Blank	0.000	22:50:01

*Sequence Aborted

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GUDMART 1144 AGENCY GOODYEAR PD
DATE 8/15/13 TIME 2250
INTOXILYZER SERIAL # 80-001858 LOCATION 1111 S LITCHFIELD RD

- (V) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (A) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(A) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(A) 4. Air blank completed.
(A) 5. Calibration check completed. Test results 0. 099 AC.
(A) 6. Air blank completed.
(A) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: 1111 S LITCHFIELD RD

Serial Number: 80-001858

Core Version: 8105.48

08/15/2013

22:50:14

Standard Lot#: 34011100A1

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	22:50:43
0.100 Cal Check	0.099	22:51:04
Air Blank	0.000	22:51:34

Preventative Maintenance
Performed -- Timer Reset