

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Will AGENCY Goodyear PD
DATE 12/7/07 TIME 2050
INTOXILYZER SERIAL # 80-001858 LOCATION Goodyear PD

- (W) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
Pour a standard alcohol concentration solution OR 100 AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (W) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (W) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (W) 4. Air blank completed.
- (W) 5. Calibration check completed. Test results 0. 099 AC.
- (W) 6. Air blank completed.
- (W) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

12/07/2007

20:50:12

Standard Lot#: 706502I

Last Changed By: W. WILL #974

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time

Air Blank	0.000	20:50:39
Cal Check	0.099	20:50:58
Air Blank	0.000	20:51:27

Operator's Signature

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

12/07/2007

18:25:42

Standard Lot#: 706502I

Last Changed By: W. WILL #974

QAS: J. ROGERS #1123

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	18:26:10
Cal Check	0.099	18:26:28
Air Blank	0.000	18:26:58

Operator's Signature

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST WILL AGENCY Goodyear PD
DATE 12/7/07 TIME 2054
INTOXILYZER SERIAL # 80-001858 LOCATION Goodyear PD

(☒) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

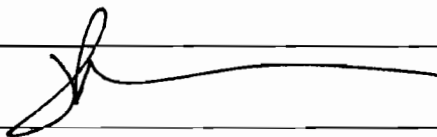
- (☒) 1. Clock time check.
(☒) 2. Date check.

OPERATIONAL TESTS

- (☒) 1. Alcohol-free subject test result 0. 000 AC.
(☒) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(☒) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(☒) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES ☒ NO ☐

COMMENTS

SIGNATURE 

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

12/07/2007

20:54:01

Standard Lot#: 706502I

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	20:54:58
Diagnostic Test	Pass	20:55:26
Air Blank	0.000	20:55:53
0.100 Cal Check	0.098	20:56:12
Air Blank	0.000	20:56:41
Subject Test	0.000	20:57:01
Air Blank	0.000	20:57:28
Five Minute Wait	ABT*	
Air Blank	0.000	20:58:06

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: GOODYEAR POLICE DEPT
Serial Number : 80-001858
12/07/2007 20:58:16

Standard Lot#: 706502I
Last Changed By: W. WILL #974

QAS: W. WILL #974
GOODYEAR PD

Operator: W. WILL #974
GOODYEAR PD

Subject: TEST,TEST
DOB: 11/11/1111
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	20:58:51
Air Blank	0.000	20:59:19

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

12/07/2007

20:59:39

Standard Lot#: 706502I

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	21:00:28
Air Blank	0.000	21:00:57

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

12/07/2007

21:01:12

Standard Lot#: 706502I

Last Changed By: W. WILL

#974

QAS: W. WILL

#974

GOODYEAR PD

Operator: W. WILL

#974

GOODYEAR PD

Subject: TEST,TEST

DOB: 11/11/1111

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:02:25
Diagnostic Test	Pass	21:02:52
Air Blank	0.000	21:03:19
0.100 Cal Check	0.099	21:03:38
Air Blank	0.000	21:04:08
Subject Test	0.086*	21:07:16
Air Blank	0.000	21:07:48
Air Blank	0.000	21:08:15
Subject Test	XXX**	21:09:11
Air Blank	0.000	21:09:44

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

12/07/2007

21:10:24

Standard Lot#: 706502I

Last Changed By: W. WILL #974

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	21:10:51
Cal Check	0.098	21:11:10
Air Blank	0.000	21:11:39

Operator's Signature

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Will AGENCY Goodyear PD
DATE 12/28/2007 TIME 1956
INTOXILYZER SERIAL # 80-001858 LOCATION GPD

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____

DPS Form Exh G-2 (Rev 05-01)

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

12/28/2007

19:56:42

Standard Lot#: 7065021

Last Changed By: W. WILL #974

QAS: W. WILL #974

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	19:57:10
Cal. Check	0.099	19:57:29
Air Blank	0.000	19:57:58

Operator's Signature

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Wiu AGENCY Goodyear PD
DATE _____ TIME _____
INTOXILYZER SERIAL # 80-001858 LOCATION Goodyear PD

- (M) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (M) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (M) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (M) 4. Air blank completed.
- (M) 5. Calibration check completed. Test results 0. 098 AC.
- (M) 6. Air blank completed.
- (M) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____