

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GUELHAULT 1144 AGENCY GOODYEAR PD
DATE 12/1/2009 TIME 0743
INTOXILYZER SERIAL # 80-00185B LOCATION GOODYEAR PD

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.101 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1144

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

12/01/2009

07:42:29

Standard Lot#: 562576

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	07:42:58
Cal Check	0.101	07:43:15
Air Blank	0.000	07:43:44


Operator's Signature

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. GUILBAULT 1144 AGENCY GOODYEAR
DATE 12/1/2009 TIME 1239
INTOXILYZER SERIAL # 80-001858 LOCATION GOODYEAR PD

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 101 AC.

Instrument is operating properly and accurately. YES K NO

COMMENTS

RANGE EXCEEDED BETWEEN REF + DEFICIENT

SIGNATURE [Signature] 1144

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

12/01/2009

12:39:16

Standard Lot#: 562576

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/2000

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:40:43
Diagnostic Test	Pass	12:41:11
Air Blank	0.000	12:41:38
0.100 Cal Check	0.102	12:41:57
Air Blank	0.000	12:42:26
Subject Test	0.000	12:42:43
Air Blank	0.000	12:43:10
Five Minute Wait	ABT*	
Air Blank	0.000	12:43:46

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

12/01/2009

12:44:00

Standard Lot#: 562576

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Subject: TEST, TEST

DOB: 01/01/2000

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	12:44:52
Air Blank	AMB**	12:45:21

*Improper Sample

**Ambient Fail

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

12/01/2009

12:45:47

Standard Lot#: 562576

Last Changed By: M. GUILBAULT #1144

OAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/2000

Sex: M Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	12:46:20
Air Blank	0.000	12:46:49

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number : 80-001858

12/01/2009

12:46:54

Standard Lot#: 562576

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Subject: TEST,TEST

DOB: 01/01/2000

Sex: M Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:47:43
Diagnostic Test	Pass	12:48:11
Air Blank	0.000	12:48:38
0.100 Cal Check	0.102	12:48:57
Air Blank	0.000	12:49:26
Subject Test	RGE*	12:49:45
Air Blank	0.000	12:50:25

*Range Exceeded

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: GOODYEAR POLICE DEPT
Serial Number : 80-001858
12/01/2009

12:50:31

Standard Lot#: 562576
Last Changed By: M. GUILBAULT #1144

OAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Subject: TEST,TEST
DOB: 01/01/2000
Sex: M Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:51:26
Diagnostic Test	Pass	12:51:53
Air Blank	0.000	12:52:20
0.100 Cal Check	0.101	12:52:39
Air Blank	0.000	12:53:09
Subject Test	0.020*	12:56:17
Air Blank	0.000	12:56:49
Air Blank	0.000	12:57:16
Subject Test	XXX**	12:57:47
Air Blank	0.000	12:58:20

*Deficient sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GURLMAN 1144 AGENCY GOODYEAR PD
DATE 12/1/2009 TIME 1258
INTOXILYZER SERIAL # 80-001858 LOCATION GOODYEAR PD

- (A) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (A) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (A) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (A) 4. Air blank completed.
- (A) 5. Calibration check completed. Test results 0. 101 AC.
- (A) 6. Air blank completed.
- (A) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1144

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001852

12/01/2009

13:58:48


Standard Lot#: 562576

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	12:59:15
Cal Check	0.101	12:59:34
Air Blank	0.000	13:00:04


Operator's Signature

INTOXILYZER 8000

Location: GOODYEAR POLICE DEPT

Serial Number: 80-001858

12/19/2009

22:24:21


Standard Lot#: 562576

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	22:24:48
Cal Check	0.101	22:25:07
Air Blank	0.000	22:25:37

 M. Guilbault #1144

Operator's Signature