

2

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GURANT AGENCY 60004th PA
DATE 12/4/12 TIME 1544
INTOXILYZER SERIAL # 80-001858 LOCATION 10701 S ESTERNA PKWY

- (A) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (S) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (S) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (A) 4. Air blank completed.
- (S) 5. Calibration check completed. Test results 0. 101 AC.
- (S) 6. Air blank completed.
- (S) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: 10701 S ESTRELLA

Serial Number: 80-001858

Core Version: 8105.48

12/04/2012

15:44:42

Standard Lot#: 06011100A1

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	15:45:12
0.100 Cal Check	0.101	15:45:33
Air Blank	0.000	15:46:02

Preventative Maintenance

Performed -- Timer Reset

C

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST 1M GULMAN AGENCY GOODYEAR PD
DATE 12/4/12 TIME 1546
INTOXILYZER SERIAL # 80-001858 LOCATION 10701 S ESTRELA PKWY

(☒) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS


- (☒) 1. Clock time check.
(☒) 2. Date check.

OPERATIONAL TESTS

- (☒) 1. Alcohol-free subject test result 0. 000 AC.
(☒) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(☒) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(☒) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 101 AC.

Instrument is operating properly and accurately. YES ☒ NO ☐

COMMENTS

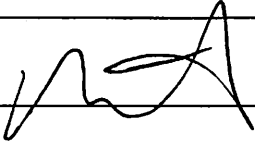SIGNATURE 

INTOXILYZER 8000
Location: 10701 S ESTRELLA
Serial Number: 80-001858
Core Version: 8105.48
12/04/2012

15:46:12

Standard Lot#: 06011100A1
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Subject: TEST,TEST
DOB: 01/01/1990
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:47:06
Diagnostic Test	Pass	15:47:41
Air Blank	0.000	15:48:08
0.100 Cal Check	0.101	15:48:29
Air Blank	0.000	15:48:59
Subject Test	0.000	15:49:19
Air Blank	IPS*	15:49:26
Air Blank	RFI**	15:49:34

*Improper Sample
**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 10701 S ESTRELLA
Serial Number: 80-001858
Core Version: 8105.48
12/04/2012

15:49:37

Standard Lot#: 06011100A1
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Subject: TEST,TEST
DOB: 01/01/1990
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:50:24
Diagnostic Test	Pass	15:50:59
Air Blank	0.000	15:51:26
0.100 Cal Check	ABT*	15:51:40
Air Blank	0.000	15:52:10

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: 10701 S ESTRELLA
Serial Number: 80-001858
Core Version: 8105.48
12/04/2012

15:52:14

Standard Lot#: 06011100A1
Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144
GOODYEAR PD

Operator: M. GUILBAULT #1144
GOODYEAR PD

Subject: TEST,TEST
DOB: 01/01/1990
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:53:01
Diagnostic Test	Pass	15:53:36
Air Blank	0.000	15:54:03
0.100 Cal Check	0.101	15:54:24
Air Blank	0.000	15:54:53
Subject Test	0.089*	15:58:01
Air Blank	0.000	15:58:33
Air Blank	0.000	15:59:00
Subject Test	XXX**	15:59:24
Air Blank	0.000	15:59:57

*Deficient Sample

**Invalid Sample

Begin new deprivation period

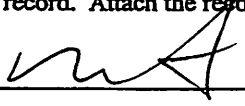
Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. GURANIT AGENCY GOOD YEAR PA
DATE 12/4/2012 TIME 1800
INTOXILYZER SERIAL # 80-001858 LOCATION 10701 SESTRILLA PKWY

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 101 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

INTOXILYZER 8000

Location: 10701 S ESTRELLA

Serial Number: 80-001858

Core Version: 8105.48

12/04/2012

16:00:14

Standard Lot#: 06011100A1

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	16:00:46
0.100 Cal Check	0.101	16:01:06
Air Blank	0.000	16:01:36

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. Gutierrez 1144 AGENCY Goodrich PD

DATE 12/27/2012 TIME 2048

INTOXILYZER SERIAL # 80-00858 LOCATION HH Sheriff's Office
10701 S ESTRELLA PKWY, 100

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 2100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 102 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 1144

INTOXILYZER 8000

Location: 10701 S ESTRELLA

Serial Number: 80-001858

Core Version: 8105.48

12/27/2012

20:48:33

Standard Lot#: 06011100A1

Last Changed By: M. GUILBAULT #1144

QAS: M. GUILBAULT #1144

GOODYEAR PD

Operator: M. GUILBAULT #1144

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	20:49:17
0.100 Cal Check	0.102	20:49:38
Air Blank	0.000	20:50:07

Preventative Maintenance
Performed -- Timer Reset