

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. ROGERS AGENCY GOODYEAR P.D.
DATE 1-21-07 TIME 1441
INTOXILYZER SERIAL # 80-001858 LOCATION GOODYEAR JUSTICE FACILITY

- ☒ 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- ☒ 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
☒ 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
☒ 4. Air blank completed.
☒ 5. Calibration check completed. Test results 0. 099 AC. LOT 536401I
☒ 6. Air blank completed. EXP 1-3-08
☒ 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] #1123

INTOXILYZER 8000

Location: GOODYEAR JUSTICE FAC

Serial Number: 80-001858

01/21/2007

14:41:19

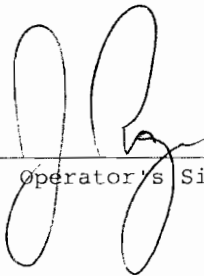
Standard Lot#: 536401I

Last Changed By: M. SLONEKER #6559

QAS: J. ROGERS #1123

GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	14:41:47
Cal Check	0.099	14:42:06
Air Blank	0.000	14:42:35

 N23

Operator's Signature