

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST K. MIYAZATO 1139 AGENCY GARDNER P.D.
DATE 2/1/2020 TIME _____
INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS


- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES NO _____

COMMENTS

SIGNATURE  1139

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

02/01/2020

04:14:40

Standard Lot#: 10818100A1

Last Changed By: W. RYAN #1100

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 209

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	04:16:07
Diagnostic Test	Pass	04:16:42
Air Blank	0.000	04:17:10
0.100 Cal Check	0.100	04:17:30
Air Blank	0.000	04:18:00
Subject Test	0.270*	04:21:07
Air Blank	0.000	04:21:42
Air Blank	0.000	04:22:09
Subject Test	XXX**	04:23:10
Air Blank	0.000	04:23:44

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

02/01/2020

00:45:27

Standard Lot#: 10818100A1

Last Changed By: W. RYAN #1100

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 209

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	00:47:11
Diagnostic Test	Pass	00:47:46
Air Blank	0.000	00:48:13
0.100 Cal Check	0.100	00:48:34
Air Blank	0.000	00:49:04
Subject Test	0.021*	00:52:11
Air Blank	IPS**	00:52:18
Air Blank	0.000	00:52:47

*Deficient Sample

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

02/01/2020

00:43:43

Standard Lot#: 10818100A1

Last Changed By: W. RYAN

#1100

QAS: K. MIYAZATO

#1139

GOODYEAR PD

Operator: K. MIYAZATO

#1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 209

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	00:44:50
Air Blank	0.000	00:45:19

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

02/01/2020

00:57:35

Standard Lot#: 10818100A1

Last Changed By: W. RYAN

#1100

QAS: K. MIYAZATO

#1139

GOODYEAR PD

Operator: K. MIYAZATO

#1139

GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M

Weight: 209

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	00:58:21
Air Blank	0.000	00:58:49

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

02/01/2020

00:38:23

Standard Lot#: 10818100A1

Last Changed By: W. RYAN #1100

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Subject: SQAP,TEST

DOB: 04/02/1973

Sex: M Weight: 209

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	00:39:58
Diagnostic Test	Pass	00:40:33
Air Blank	0.000	00:41:01
0.100 Cal Check	0.100	00:41:22
Air Blank	0.000	00:41:51
Subject Test	0.000	00:42:14
Air Blank	ABT*	00:42:38
Air Blank	0.000	00:43:06

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: 11 N 145TH AVE

Serial Number: 80-001858

Core Version: 8105.48

02/01/2020

00:21:43

Standard Lot#: 10818100A1

Last Changed By: W. RYAN #1100

QAS: K. MIYAZATO #1139

GOODYEAR PD

Operator: K. MIYAZATO #1139

GOODYEAR PD

Test g/210L Time

Air Blank 0.000 00:22:25
0.100 Cal Check 0.100 00:22:46
Air Blank 0.000 00:23:15

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Kory Miyazato AGENCY Goodyear
DATE 2/1/2020 TIME 0021
INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145 TH AVE

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 100 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Kory Miyazato 1139

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
02/01/2020 01:02:43

Standard Lot#: 10818100A1
Last Changed By: W. RYAN #1100

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	01:03:13
0.100 Cal Check	0.100	01:03:34
Air Blank	0.000	01:04:03

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST K. MIYAZATO 1139 AGENCY Goodyear P.D.
DATE 2/29/20 TIME 1548
INTOXILYZER SERIAL # 80-001858 LOCATION 11 N 145TH AVE

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 099 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE K. Miyazato 1139

INTOXILYZER 8000
Location: 11 N 145TH AVE
Serial Number: 80-001858
Core Version: 8105.48
02/29/2020 15:48:39

Standard Lot#: 10818100A1
Last Changed By: W. RYAN #1100

QAS: K. MIYAZATO #1139
GOODYEAR PD

Operator: K. MIYAZATO #1139
GOODYEAR PD

Test	g/210L	Time
Air Blank	0.000	15:49:16
0.100 Cal Check	0.099	15:49:37
Air Blank	0.000	15:50:07

Preventative Maintenance
Performed -- Timer Reset